

# idi

Image Diagnostics, Inc.

## *MDS*

### Mobile Display System



## *Installation & Operator Manual*

Made in the USA

# Installation & Operator Manual for MDS

The text of this manual was originally written, approved and published by the manufacturer in English.

## OVERVIEW

This manual pertains to the specified devices only and does not intended to replace or substitute for certified training in the application of this equipment. The device is intended for qualified medical personnel who have been trained in the use of medical equipment.

Functional capabilities and operation of the equipment described herein can be employed in a variety of diagnostic, therapeutic, and surgical applications.

## OWNER RESPONSIBILITIES

The owner of this device is responsible to ensure system compatibility, the qualifications of the operator and maintenance personnel. The operator must be properly trained and have obtained credentials from the appropriate authorities.

This equipment must be installed in an area provided with the proper electrical power.

The owner of this device is responsible for verifying continued compliance with all applicable regulations and standards. Consult local, state, federal and/or international agencies regarding specific requirements and regulations applicable to the use of this equipment.

Image Diagnostics, Inc. certifies only the equipment. After-sale operating practices and safety are the responsibility of the owner and operator. Image Diagnostics, Inc. assumes no liability or responsibility for after-sale operating or safety practices; nor can it be responsible for personal injury or damage resulting from misuse.

Never make modifications or adjustments to the equipment unless directed by a qualified Image Diagnostics representative. This equipment, when properly assembled, meets US federal regulations and international standards. Unauthorized modifications to the equipment may impact adherence to these standards and make the equipment unsafe to operate.

## CUSTOMER SUPPORT

Image Diagnostics will make available, on request, circuit diagrams, component part lists, calibration instructions, or other information which will assist the user's appropriately qualified technical personnel to repair those parts of equipment which are designated by the manufacturer as repairable.

For technical assistance, call IDI at (978) 829-0009 or send fax to (978) 829-0027. Be prepared to give the complete model and serial number found on the data label on the base of the MDS at the time of contact.

# Installation & Operator Manual for MDS

## Table of Contents

|   |           |
|---|-----------|
| <b>1. SYMBOL IDENTIFICATION.....</b>                          | <b>1</b>  |
| <b>2. INTENDED USE &amp; ESSENTIAL PERFORMANCE .....</b>      | <b>2</b>  |
| <b>3. SAFETY INSTRUCTIONS .....</b>                           | <b>2</b>  |
| <b>4. SAFETY HAZARDS .....</b>                                | <b>3</b>  |
| <b>5. EMC (Electromagnetic Compatibility) STATEMENT .....</b> | <b>5</b>  |
| <b>6. INSTALLATION INSTRUCTIONS .....</b>                     | <b>6</b>  |
| 6.1. Mount Two Main Monitors.....                             | 7         |
| 6.2. Release Monitor Counterbalance .....                     | 9         |
| 6.3. Shelf Adjustment .....                                   | 10        |
| 6.4. Clean Unit .....   | 11        |
| 6.5. Electronics Installation .....                           | 11        |
| <b>7. OPERATOR INSTRUCTIONS .....</b>                         | <b>12</b> |
| 7.1. Moving the MDS Unit .....                                | 12        |
| 7.2. Moving Monitor Positions .....                           | 13        |
| <b>8. MAINTENANCE, SERVICE &amp; REPAIR .....</b>             | <b>14</b> |
| 8.1. Recommended Periodic Performance Checks .....            | 14        |
| 8.2. Service & Repair Statement .....                         | 15        |
| <b>9. TROUBLESHOOTING .....</b>                               | <b>15</b> |
| <b>10. DISPOSAL OF COMPONENTS .....</b>                       | <b>15</b> |
| <b>11. SPECIFICATIONS/PRODUCT DATA .....</b>                  | <b>16</b> |
| <b>12. WARRANTY .....</b>                                     | <b>17</b> |

# Installation & Operator Manual for MDS

## 1. SYMBOL IDENTIFICATION



Attention! Consult accompanying documents. Failure to follow these instructions could cause serious personal injury or damage to equipment. Consult all accompanying documents.



Warning! Information or instructions shown near this symbol must be adhered to in order to prevent a potentially hazardous situation which if not avoided, could result in death, personal injury or damage to the equipment.



Recyclable material.



Protective Ground. This is the common tie point between the AC Electrical Power Cord Ground, Frame Ground, and Controller Ground.



SGS North America, Inc. Testing Service.



European Authorized Representative:

Advena Ltd. Tower Business Centre, 2nd Flr.,  
Tower Street, Swatar, BKR 4013 Malta



Model of Video Integration System.



Serial Number of Video Integration System.



Indication of European Conformity for sale in the European Economic Area (EEA).



Date of manufacture of the Video Integration System.



Location where the Video Integration System was manufactured.



Alternating Current (AC).

## Installation & Operator Manual for MDS

### 2. INTENDED USE & ESSENTIAL PERFORMANCE

The MDS (Mobile Display System) is a mobile solution for video display and image management. This system features flexibility with its video display positioning combined with video signal routing and multiple image management options. The MDS is designed to configure and control images from C-arm fluoroscopy, endoscopy, patient monitoring, etc.

### 3. SAFETY INSTRUCTIONS



Review the SAFETY HAZARDS and OPERATING INSTRUCTIONS before operating MDS

- ✓ All persons using this equipment must fully understand its operation instructions, emergency procedures, capabilities including total range of motion and be aware of all potential safety hazards.
- ✓ This manual should be accessible to all personnel installing, operating, or servicing this equipment.
- ✓ Only a qualified technician may install or service this equipment.



***Failure to follow safety precautions may result in serious injury to patient or user or damage to equipment.***

Comments and questions regarding safety should be addressed to:



Customer Support  
Image Diagnostics, Inc.  
310 Authority Drive  
Fitchburg, MA 01420 USA






Or call IDI at (978) 829-0009 or Toll Free (877) 304-5434.  
The fax number for IDI is (978) 829-0027.

## Installation & Operator Manual for MDS

### 4. SAFETY HAZARDS

It is imperative that all personnel operating the MDS be familiar with the equipment's operation, transport and all documentation supplied by IDI. The MDS system is intended to be used in typical clinical or research environments in accordance with national medical standards. When transporting the MDS from room to room, it is imperative that the device be moved while in its storage configuration with the security strap in place and the arms folded into the unit so that the magnetic stops are engaged. The MDS should be plugged into the appropriate medical grade electrical outlet, not to exceed the rated amp values as stated on the MDS serial number label.

| Safety Hazard Level   | Potential Consequences with Use   |
|---|---|
|  <b>DANGER</b>   | Indicates an <i>imminently</i> hazardous situation which, if not avoided, will result in death or serious injury.                     |
|  <b>WARNING</b>  | Indicates a <i>potentially</i> hazardous situation which, if not avoided, could result in death or serious injury.                    |
|  <b>CAUTION</b> | Indicates a <i>potentially</i> hazardous situation which, if not avoided, may result in minor or moderate injury or equipment damage. |



#### **WARNING!**

**Do not jump any curbs or steps over 3/8" in height with this equipment. Approach all obstacles slowly. Operator must use a ramp to allow the casters to roll up and over the vertical edge.**



#### **WARNING!**

**It is the user's responsibility to make sure the equipment is safe and operates properly prior to use.**



#### **WARNING!**

**Do not modify this equipment without authorization of the manufacturer.**

## Installation & Operator Manual for MDS



### ***CAUTION***

**Caution should be used when moving the system over uneven or sloped surfaces.**



### ***CAUTION***

**Confirm that the monitor arm heights are appropriate for the clinical application, otherwise store in transportation mode.**



### ***CAUTION***

**Prior to each clinical placement, confirm that there is no vertical drift on the horizontal arms and that the swivel and tilt mechanisms are functioning correctly.**



### ***CAUTION***

**Injury may occur if the operator inadvertently runs into the extended monitor arms. Be aware of the arm's location at all times during the installation and while in**



### ***CAUTION***

**If upper Locking Collars are loose, they may inadvertently drop and prevent free vertical arm**



### ***CAUTION***

**Do NOT store liquids on the unit above any electrical devices.**

## Installation & Operator Manual for MDS

### 5. EMC (Electromagnetic Compatibility) STATEMENT

Portable and Mobile RF Communications Equipment can affect Medical Electrical Equipment including the ones used with the MDS. Use special precautions regarding EMC when these Mobile Display Systems are installed, operated and maintained.

The use of accessories, transducers and/or cables other than those specified, with the exception of those sold by the manufacturer as replacement parts for internal components, may result in increased emissions or decreased immunity of the equipment or system.



**WARNING:** If MDS is used adjacent to or stacked with other medical electrical equipment, the equipment and the MDS should be observed to verify normal operation in the configuration in which it will be used.

**NOTE:** The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.



### 6. INSTALLATION INSTRUCTIONS

#### Scope:

This section describes the process of preparing the MDS for Video Integration. The MDS monitor arms are counter-balanced with a series of steel plates (behind the rear cover) that are sized to accommodate the weight of a variety of medical monitors. Each MDS system is factory balanced to the specifications based on the weight of the Monitors supplied with the system. The Monitors are removed for shipment and the weights are secured in place via upper and lower Locking Collars.



#### ***WARNING!***

**Loosening the Shipping Locking Collars without the monitors being installed will create a severe imbalance and uncontrolled rapid arm movement!**



#### ***CAUTION!***

**When mounting or removing the monitors, confirm that the vertical travel lock is engaged and the upper/lower collar locks are secured.**

**Note:** The upper and lower Locking Collars are used to secure the arms for shipping only. After relocating the Collars, the Vertical Travel Lock Hooks should ONLY be utilized during transportation of the unit.

**Note:** The original factory balance weight is noted on the face of the VESA Monitor Support Bracket and the Face Balance Plates. If Monitors are upgraded to smaller or larger sizes, it will be necessary to rebalance the Monitor Arms with the new monitor weight. Additional plates can be ordered from IDI if needed. Please contact customer service prior to changing Monitors. It is recommended that two persons assist in the mounting/un-mounting of Monitors.



#### ***WARNING!***

**The mounted monitors can be considered as suspended mass which can fall and cause injury if they are not properly mounted.**


# Installation & Operator Manual for MDS

## 6.1. Mount Two Main Monitors.

**Tools:** 5/32" Hex Key, Torqueing Phillips (Cross) Screwdriver, #222 Loctite and Orange Torque Seal.

### Procedure:

- 6.1.1. Using a 5/32" Hex Key, remove the MDS Back Panel by removing all the upper and lower Socket Head Cap Screws.
- 6.1.2. Locate and confirm the upper and lower rail Locking Collars are secure and in place against the top and bottom Counterweight Bracket Flanges as shown in the photos below.

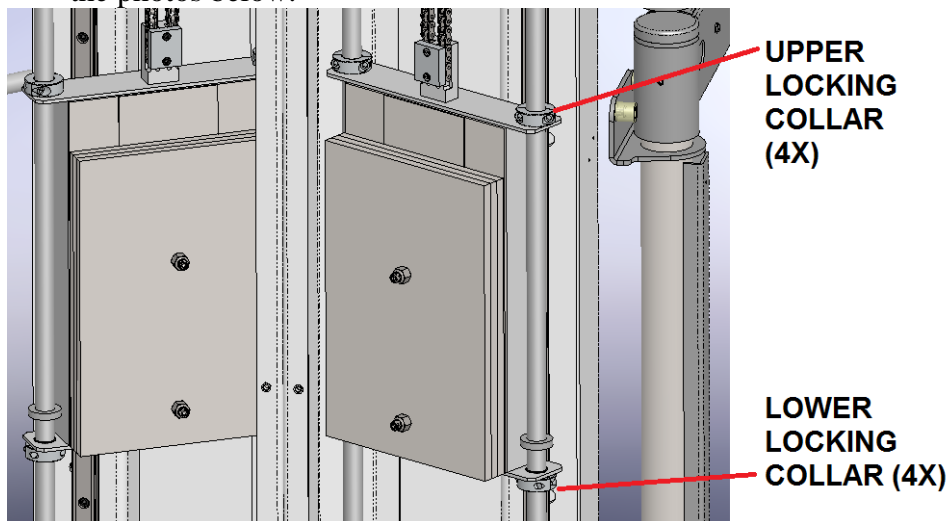

### **WARNING!**

**If the Locking Collars are found to be loose, secure with the Hex Key before proceeding.**

**NOTE:** The upper and lower Locking Collars are used to secure the Arms for shipping only, and after they are relocated the Vertical Travel Lock Hooks should **ONLY** be utilized during transportation of the unit.

- 6.1.3. Unpack the two main Monitors carefully and lay them face down on a protective surface.
- 6.1.4. Locate and remove the four mounting screws on the back panel of the Monitor.

## Installation & Operator Manual for MDS

- 6.1.5. Apply #222 Loctite to two of the screws and partially install the upper two screws as to allow enough of a gap below the screw heads so they can easily slide into the top slots of the VESA mounting plates on the MDS arm while still being able to support the weight of the Monitor. See photo below.



### **SLOTS ON MDS ARM MONITOR MOUNTING PLATE**

- 6.1.6. With assistance from another individual, lift the Monitor with the partially installed upper screws and slide the screws into the upper slots of the VESA Mounting Plates on the MDS arm.
- 6.1.7. Apply #222 Loctite to the two remaining screws and install them into the bottom holes of the VESA Mounting Plates on the MDS arm. Torque all four screws to 23lbf-in. (2.6N-m).
- 6.1.8. Apply orange torque seal to the heads of the four monitor mounting screws.
- 6.1.9. Connect the power cable and the video cable to the appropriate port input.
- 6.1.10. Repeat steps 6.1.2 through 6.1.9 with the second Monitor.


# Installation & Operator Manual for MDS

## 6.2. Release Monitor Counterbalance

**Tools:** 3/16" Hex Key.

### Procedure:

- 6.2.1. Once both main Monitors are mounted and secure, the counterweight Locking Collars are ready to be relocated from their storage positions.
- 6.2.2. With a 3/16" Hex Key, loosen the lower Locking Collars and lower them to the bottom of the slide rail. Loosen and raise the upper Locking Collars to the top of each slide rail and retighten as shown below.

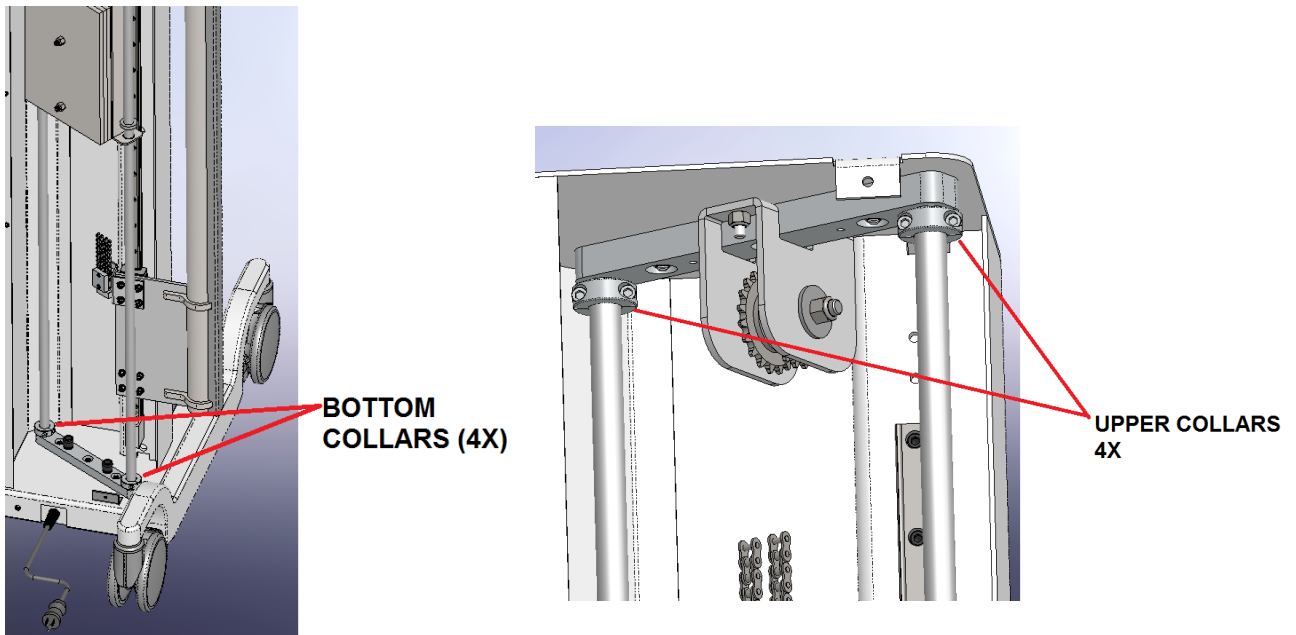

**NOTE:** Arms will not move vertically until Vertical Travel Lock Hooks are disengaged.

## Installation & Operator Manual for MDS

- 6.2.3. Release the red Vertical Travel Lock Hooks shown below and each monitor arm should lower and raise with minimal physical effort.



- 6.2.4. If the monitor arms drift up or down easily or drift in one direction or another, the unit may need to be rebalanced. Contact IDI Customer Service for procedure steps.


**NOTE:** There are inner and outer locking collars for each balance plate.

### 6.3. Shelf Adjustment

**Tools:** 3/16" Hex Key, a Level and an assistant.

#### Procedure:

- 6.3.1. Remove all devices off the Shelf that is to be repositioned.  
6.3.2. Have an assistant hold onto the sides of the Shelf to be moved.

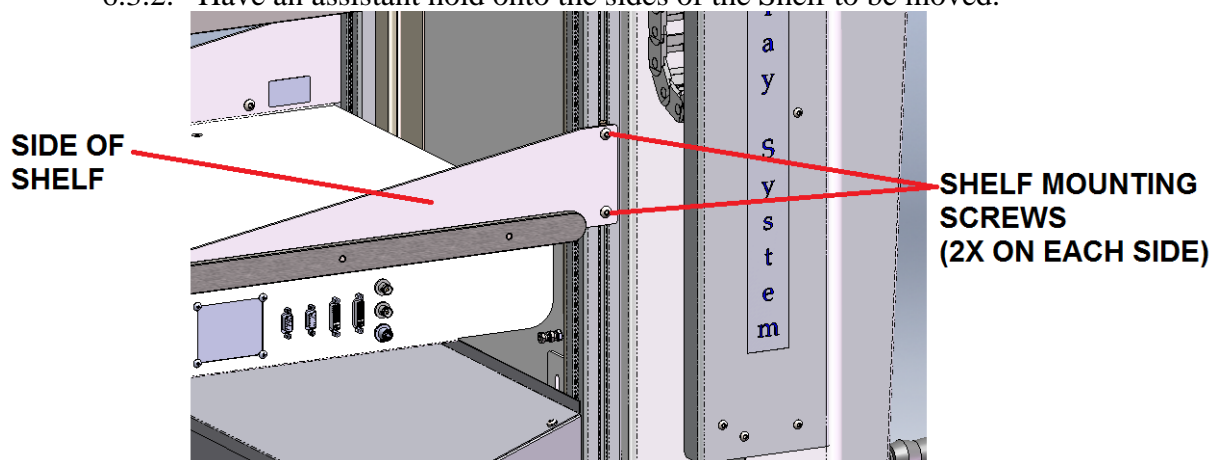

- 6.3.3. Using a 3/16" Hex Key, loosen the Button Head Socket Screws (2 on each side of Shelf) just to the point where it can rotate freely but **DO NOT** remove.  
6.3.4. The assistant can now slide the Shelf up or down to a desired location.

## Installation & Operator Manual for MDS

- 6.3.5. Using a Level, measure the Shelf to ensure that it is level and then re-tighten all four Screws.
- 6.3.6. Replace Devices onto the shelf and tuck power or video cords into the side Channels of the Cart or wrap around cable hooks provided on sides of second shelf from the bottom.

### 6.4. Clean Unit

**Tools:** Soft Cloth and one of the following Approved Cleaners:

- Sodium hypochlorite (generic household bleach) in a solution of 5.25% sodium hypochlorite diluted between 1:10 and 1:100 with water.
- Alcohol (generic).
- Envirocide®

#### Procedure:

- 6.4.1. Unplug the MDS system prior to cleaning the unit.
- 6.4.2. Wipe clean the surfaces of the MDS system with a diluted mixture of mild detergent (listed above) and water using a soft cloth.

**NOTE:** For cleaning Video Routers and Monitors, please consult corresponding literature for these devices. NEVER use alcohol to clean the monitors.

### 6.5. Electronics Installation

#### Procedure:

- 6.5.1. All electronic devices are to be installed according to their perspective manufacturer's IFU (Information for use) documentation.

## 7. OPERATOR INSTRUCTIONS

### 7.1. Moving the MDS Unit

**Tools:** Security Strap and an assistant.



#### **WARNING!**

**Always use a minimum of two persons while transporting the MDS unit to aid in maneuverability and to aid with visibility.**

#### Procedure:

- 7.1.1. Lower both Arms of the MDS Unit to a comfortable height to ease in the installation of Monitor covers.
- 7.1.2. Fold the Arms into themselves so that the stainless-steel Covers are facing towards the outside as shown to the right.
- 7.1.3. The MDS arms are equipped with rotational stops with built in magnets that help prevent inadvertent arm movement. Confirm that Arms are pushed in and Magnets are engaged.
- 7.1.4. Lift the arms so that the base of the monitor clears the upper most Device placed on the Top Shelf and rotate the face of one of the Monitors inward and gently lower the Monitor on to the top shelf. Repeat the process for the other Monitor.
- 7.1.5. With both Monitors facing inward, place the Security Strap over the top of each Monitor Handle as shown to the right and secure snugly.



**SECURITY STRAP**



#### **CAUTION!**

**The user should confirm that the arms are folded in as far as possible, and the magnets are engaged, in order to prevent unintended arm swing.**

## Installation & Operator Manual for MDS

- 7.1.6. Disengage caster motion locks on all four casters by pushing up on caster lock levers with top of foot as shown on the right.
- 7.1.7. MDS unit can now be transported by two individuals using the handles located on the sides and back to maneuver the unit.
- 7.1.8. Once the MDS unit is relocated to its final position, all four casters should be motion locked by stepping down on the caster lock pedal until they are fully engaged.



**UNLOCKING  
CASTER MOTION**

### **7.2. Moving Monitor Positions**

#### **Procedure:**

- 7.2.1. To move monitor position, grab the monitor handle just below the monitor.
- 7.2.2. Push and pull monitor to desired location within the limits of the articulating arms they are mounted to.
- 7.2.3. To tilt monitor position downward, hold the top of the monitor mount and push the monitor handle away from the operator.
- 7.2.4. To tilt monitor position upward, hold the top of the monitor mount and pull the monitor handle away toward the operator.



#### **CAUTION!**

**Vertical travel of monitor is counterbalanced, so there should be no excessive force required to move monitors up and down. If excessive force is required, please check that the transport lock brake is unlatched and if this does not resolve the issue please contact IDI service for further troubleshooting.**

- 7.2.5. To raise the monitor position, grab the monitor handle and push upward.
- 7.2.6. To lower monitor position, grab the monitor handle and pull downward.



## 8. MAINTENANCE, SERVICE & REPAIR

All maintenance procedures should be done by an experienced and qualified technician with demonstrated knowledge and skills (electrical and mechanical) in the service of medical equipment.

- ✓ This individual must have access to this manual and the proper tools.
- ✓ Lubrication of this device is *not* required.

### 8.1. Recommended Periodic Performance Checks

|        |   |
|--------|---|
| Daily  | Inspect all external cables and electrical components for wear and damage. Damaged cables must be replaced promptly. The MDS uses a medical grade power cord which is not user serviceable. Replacement must be performed only by a qualified service technician. |
| Weekly | Move the monitors through their full range of motion to verify that they are not worn or damaged.   |

# Installation & Operator Manual for MDS

## 8.2. Service & Repair Statement

Only qualified personnel should perform repairs on this equipment. Please read this entire document before performing any diagnostics or repairs. Some procedures listed require this device to be energized while repairs are performed; please exercise extreme caution while working with electrical components. Always exercise appropriate lockout/tag out procedure while performing any diagnostics and service on the unit.

## 9. TROUBLESHOOTING

| Problem/Symptom                                       | Possible Cause      | Remedy   |
|---|---------------------|--|
| 1. MDS generates or receives electrical interference. | 1. Incompatibility. | 1. Verify that the equipment is the cause by turning the system or individual components off and on.<br>2. Use only high quality cables to avoid electrical noise. |

## 10. DISPOSAL OF COMPONENTS



The IDI MDS base and arms are made up of mostly steel and aluminum parts which are easily recycled. The casters are made of mostly plastic. It is recommended that some components be disassembled before disposal for recycling.

## 11. SPECIFICATIONS/PRODUCT DATA

### Type of Equipment:

- Class I (as defined by IEC 60601-1, 3rd Edition)
- CENELEC EN 60601-1, ANSI/AAMI ES60601-1, and IEC 60601-1, 3rd Edition)

### Electrical:

- Supply Voltage:
  - ★ 120±5% Vac 60Hz through NEMA 5-15P hospital grade plug.
  - ★ 230±10% Vac 50Hz through country specific plug.
- Output Voltage: 120 VAC 60Hz (1800VA) or 230 VAC 50Hz (1800VA).
- Current Rating: Less than 12 Amps @ 120 Vac or 7 Amps @ 230 Vac.
- Isolation transformer limits leakage current to 50µA.
- Isolation transformer is certified to meet Medical Safety Standards ANSI/AAMI ES60601-1.

### Environmental:

- Operating Temperature Range: 41°F to 95°F (5°C to 35°C)
- Operating Humidity Range: 10% to 85% relative humidity, noncondensing.
- Operating Pressure Range: 500 hPa to 1060 hPa.
- Transport & Storage Temperature Range: 32°F to 113°F (0°C to 45°C).
- Transport & Storage Humidity Range: 10% to 85% relative humidity, noncondensing.
- Transport & Storage Pressure Range: 500 hPa to 1060 hPa.

Not suitable for oxygen rich environment.

Not suitable for flammable anesthetic mixtures.

### Product Features Included With MDS:

- Thermal protector AC input module.
- Medical Grade output strip for Monitor & Ancillary power.
- Individual cables for monitor power and video input.

## 12. WARRANTY

**Warranty details for IDI Products can be obtained directly from Image Diagnostics, Inc.**



Image Diagnostics, Inc.  
310 Authority Drive  
Fitchburg, MA 01420 USA



Or call IDI at (978) 829-0009 or Toll Free (877) 304-5434.  
The fax number for IDI is (978) 829-0027.