

Scan Here for Operator Manual

Scroll to ilex D

idi

Image Diagnostics, Inc.

Ilex D Programming Q-Card



WARNING!

Read operator manual prior to use

Power On Monitor:

Monitor is equipped with power cord. Plug power cord into hospital grade outlet. Press "ON/OFF" switch on rear bottom of monitor. Lights will display on front of monitor.


Press power button on menu buttons if required.

Plug in video sources to be connected to monitor

Cables are labeled as shown in table below

Route desired cables to the respective device per table

Video Source	Input Port	Label Color on Cable
C-Arm Live	DVI1 or DP1(if applicable)	Green
C-Arm Ref	DVI3 or DP2(if applicable)	Black
IVUS/Hemo/Vitals/etc	DVI2	Red
IVUS/Hemo/Vitals/etc	DVI4	Blue

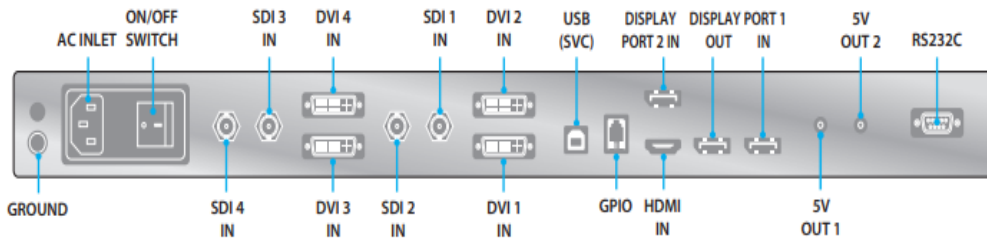

*Graphic from FSN Doc. # FSN2032 Rev. 2/20




HDMI vs DP cable, shown.

Select Layout or "PIP Mode"


To change layouts press PIP button on remote

Each press of the PIP button will cycle to the next layout

As shown, Single, PIP, PBP, Triple, Quad

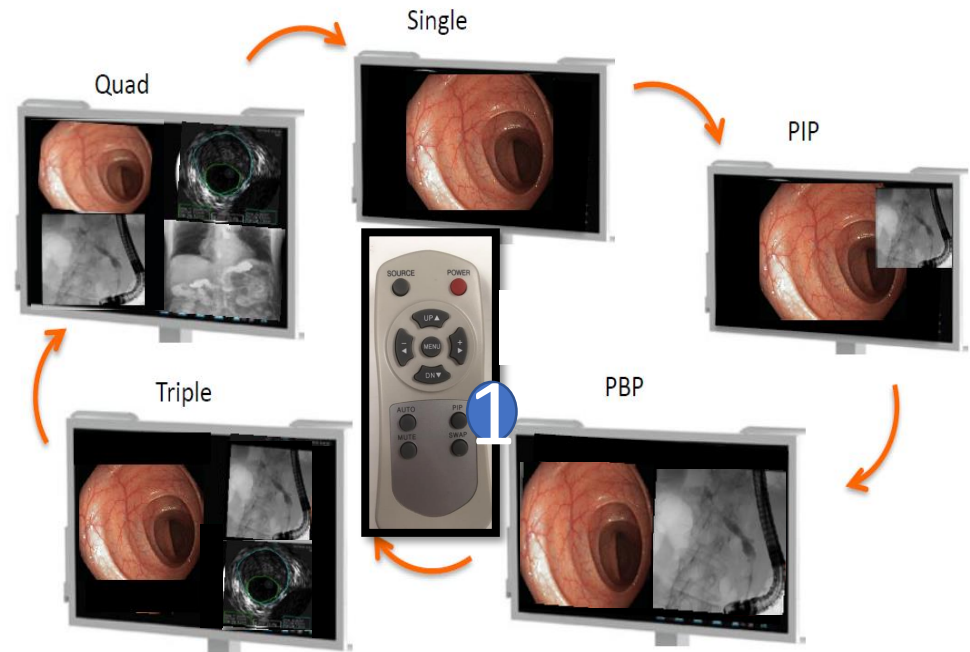


Typical Layouts

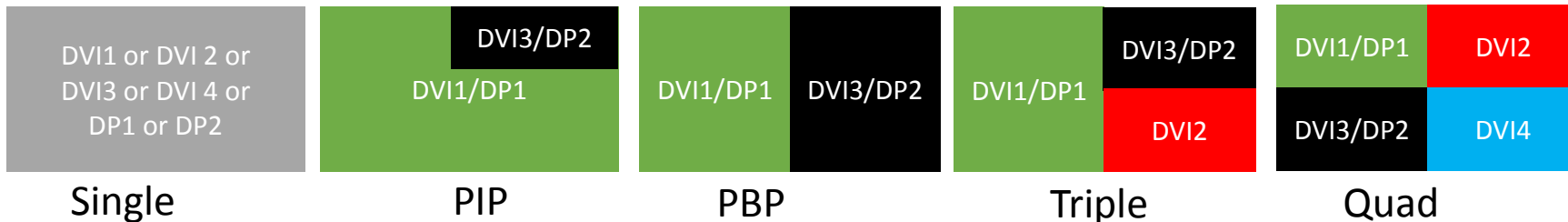


Ilex D Programming Q-Card

1. Press Source (Green Window will appear)
2. Press Up/Down to select desired input
3. "+" (right arrow) to enter selected input


Changing Sources

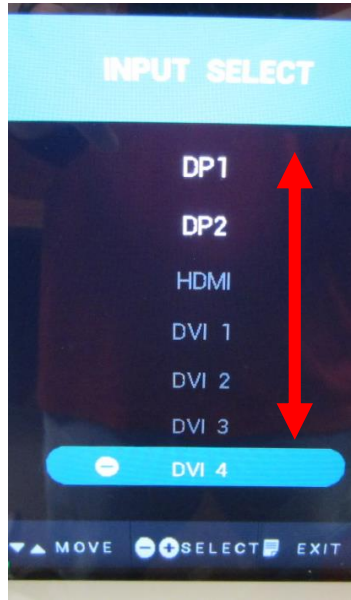

Active window will always be framed by a green boundary line, when source button is pressed

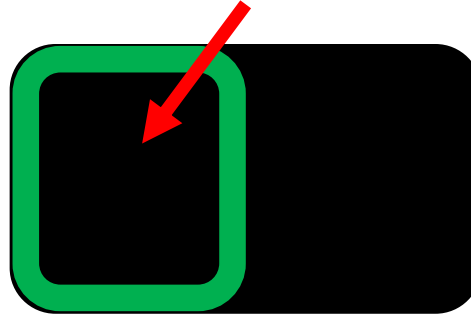



Press Source again, to switch to different active window

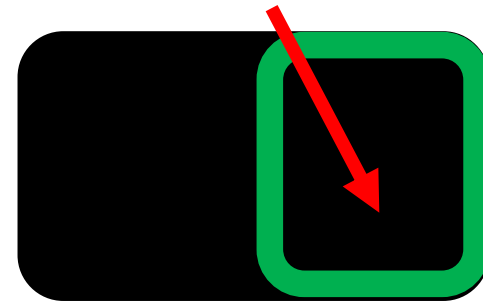

Adjusting Height of the Monitor:

The monitor height is adjustable.

To adjust the height loosen, turn counterclockwise, the brake mechanism on the rear of the column. Lift up or push down on the rear column handles. Then tighten brake in place once desired height is reached.

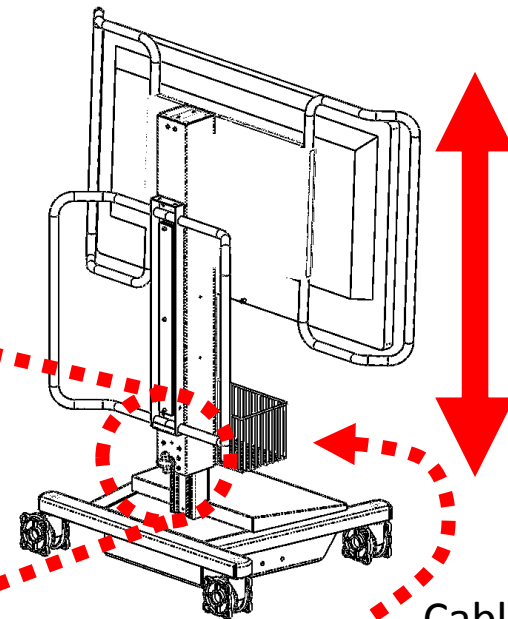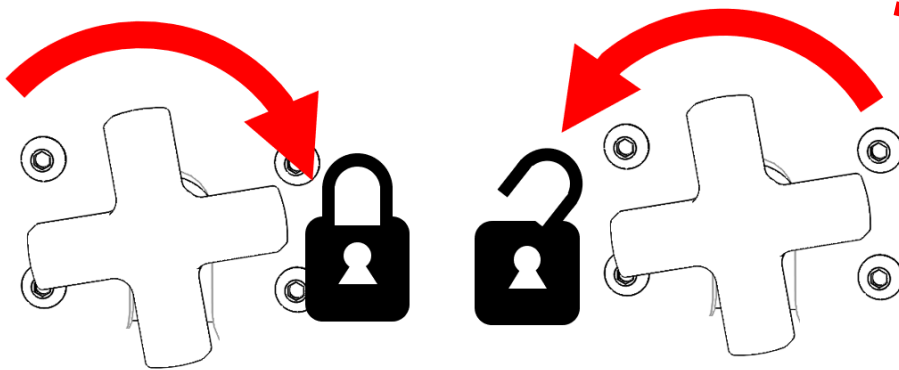

Cable Storage

Note: In some cases SDI1 will show up, to go back to DVI1 navigate to SDI1 then press "-" to switch back to DVI1 then press "+" to accept change. This applies to all 4 DVI/SDI channels.

