


# FSN Monitors Cleaning Q-Card

idi

Image Diagnostics, Inc.

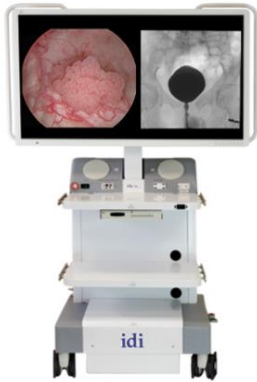

- Turn off monitor and unplug power cord.
- Take care not to damage or scratch the front filter or LCD panel.
- Be careful with rings or other jewelry that can touch the front filter.
- Do **NOT** apply excessive pressure on the monitor screen.
- Do **NOT** apply or spray liquid directly to the front filter, panel or cabinet as excess liquid may cause damage to internal electronics. Instead, apply the liquid to the cleaning cloth.
- Follow your hospital or clinical protocol for the handling of blood and body fluids.
- The display is not disinfected nor packed in sterile environment.
- Follow your hospital or clinical protocol in instances that need the display disinfected prior to installation.
- Cleaning guidelines for displays must only be carried out by medical professionals and must not be handled by patients.
- For complete cleaning instructions please reference the monitor's user manual provided with the unit.

## FSN Monitors Approved Cleaners



- Bohle Glass Cleaner
- Cabinet Only - Precise Hospital Foam Cleaner disinfectant
- Cabinet only - Virex Ready to use disinfectant cleaner
- Household bleach (generic sodium hypochlorite, solutions of 5.25% sodium hypochlorite, solutions of 5.25% sodium hypochlorite diluted with water between 1:10 and 1:100)
- Incidin Foam (Ecolab)
- Isopropyl alcohol with concentration <5%
- Klear Screen
- Microzid
- Mild Detergent
- Misty Clear Lemon 10 Disinfectant
- Screen TFT (Kontakt Chemie)
- ZEP Heavy Duty glass & all surface Cleaner

## Do Not Use List



- Acid
- Alcohol/solvents at higher concentration >5%
- Cloth with steel thread
- Detergents with abrasives
- Detergents with ammonia
- Detergents with fluoride
- Sponge with abrasives
- Steel blades
- Steel wool
- Strong alkalis lye, strong solvents
- Synthetic (polyester) cloth