

LG Monitors Cleaning Q-Card



- Turn off monitor and unplug power cord.
- Take care not to damage or scratch the front filter or LCD panel.
- Be careful with rings or other jewelry that can touch the front filter.
- Do **NOT** apply excessive pressure on the monitor screen. Use less than 1N (0.2lbs) of force.
- Do **NOT** apply or spray liquid directly to the front filter, panel or cabinet as excess liquid may cause damage to internal electronics. Instead, apply the liquid to the cleaning cloth.
- Cleaners could cause serious damage if it leaks inside the monitor while cleaning.
- Follow your hospital or clinical protocol for the handling of blood and body fluids.
- The display is not disinfected nor packed in sterile environment.
- Follow your hospital or clinical protocol in instances that need the display disinfected prior to installation.
- Cleaning guidelines for displays must only be carried out by trained medical professionals and must not be handled by patients.
- For complete cleaning instructions please reference the monitor's user manual provided with the unit.

LG Monitors Approved Cleaners



- 0.9 % NaCl Solution
- Ethanol 70%
- 70% isopropyl alcohol or Isopropanol
- Cidex® (2.4% glutaraldehyde solution)

Do Not Use List



- Acid
- Alcohol/solvents at higher concentration >5%
- Alkaline cleaners or other such solvents
- Benzene
- Cloth with steel thread
- Detergents with abrasives
- Detergents with ammonia
- Detergents with fluoride
- Sponge with abrasives
- Steel blades
- Steel wool
- Strong alkalis lye, strong solvents
- Synthetic (polyester) cloth
- Thinner