




Manufacturer of Quality Medical Products

Image Diagnostics, Inc.

IDI 1000F Series Monitor Suspension System Model IDI 1000F-1 thru 1000F-6

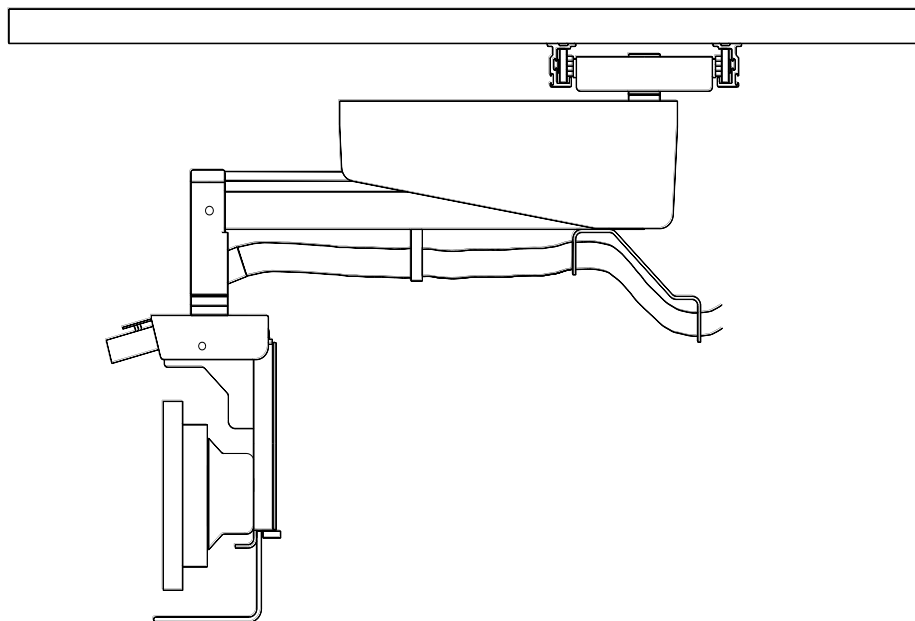

With OTC Adapter Installation and Maintenance Guide

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use with OTC Adapter

The text of this manual was originally written, approved and published by the manufacturer in English.

The information in this manual is subject to change without notice.

CONTENTS

<u>SECTION</u>	<u>TITLE</u>
A.....	INTRODUCTION, PRODUCT DATA, AND OPERATION
B.....	SYMBOLS
C.....	GENERAL SAFETY
D.....	SAFETY HAZARDS
E.....	INSTALLATION
F.....	CLEANING
G.....	MAINTENANCE
H.....	CUSTOMER SUPPORT
I.....	DISPOSAL GUIDELINES
J.....	APPROVALS

A. INTRODUCTION

The IDI 1000F Flat Panel Suspension System transports monitors used in medical imaging procedures. The system features a counterbalanced arm with vertical travel and internal cable routing. A dual-axis flat panel monitor mount permits independent panel mounting and positioning. The mobile package includes clip-mounted bridge rails.

PRODUCT DATA

Package Includes:

Mobile gantry and counterbalanced swing arm with 42" sweep radius, 25" vertical travel and $\pm 180^\circ$ rotation

Anodized aluminum bridge rails with *Easymount* clip system

Flat panel array with $\pm 180^\circ$ rotation

Array includes display panel mounts with $\pm 10^\circ$ tilt

Complete installation package; meter box mounts, all mounting hardware and cable management components; brackets, covers and 25' "flexhaust" hose.

OPERATION

The monitors are moved along the overhead rails, positioned vertically, and rotated manually using the handle.

B. SYMBOLS



Attention, consult accompanying documents.
Failure to follow these instructions can cause accidents resulting in serious injury to patient, user, and damage to equipment.




Caution, risk of electrical shock



Protective Ground
This is the common tie point between the AC monitor cord grounds, frame ground, and service (main) ground.



Alternating Current

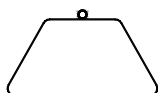

Weight Limit



Recycle
Some of the materials can be recycled rather than discarded.

C. GENERAL SAFETY



**Attention, consult accompanying documents.
Failure to follow these instructions can cause accidents
resulting in serious injury to patient, user, and damage to
equipment.**

Only qualified persons may install, operate or maintain this equipment.

Installation of this unit must adhere to applicable codes and authorities having any jurisdiction over this installation. The customer's architect or engineer is responsible for assuring that the structural support plans comply with all applicable codes and regulations.

The unit should be used only in rooms that comply with state, federal and local recommendations concerning electrical safety when used in medical installations. All electrical connections shall be done by licensed/approved electrician per national electric codes.

- CAUTION:**
- Always switch off mains before performing nonelectrical tests or maintenance.
 - Many positions exist where equipment or patient collision may occur. Care must be exercised during equipment positioning to avoid a patient or equipment collision.

Changes and additions to the equipment may be performed only by an authorized representative. These changes must conform to regulations and accepted standards of good practice. To prevent defeat of the built-in safety mechanisms, changes must be submitted in writing to the manufacturer for review.

Use only parts specified by Image Diagnostics, Inc. when repairing or servicing this equipment.

This equipment is intended to only be used with Image Diagnostics, Inc. rails.

D. SAFETY HAZARDS

Installers and Operators using this equipment should understand the safety issues and operating instructions provided.

Comments and questions regarding safety should be addressed to:

**Customer Support
Image Diagnostics, Inc.
310 Authority Drive
FITCHBURG, MA 01420
USA**

Or call IDI at (978)829-0009 or send fax to (978)829-0027
Or toll free 877-304-5434

SAFETY HAZARDS ALERTS

Alert	Circumstances for use
DANGER	Indicates an <i>imminently</i> hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Indicates a <i>potentially</i> hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Indicates a <i>potentially</i> hazardous situation which, if not avoided, may result in minor or moderate injury or equipment damage.

E. INSTALLATION

UNPACK




No special handling is required for unpacking this equipment at the site.
Conventional shipping materials are used.
Recycle or disposal of shipping material per local regulations

INSTALL ADAPTER CARRIAGES *(Refer to the OTC Rail Adapter Assembly Drawing)*

1. Install the two adapter carriages (item 1) directly to the OTC rails: remove the lower rail wheels (item 4) from the adapter carriages, hook adapter carriages over OTC rails, and reinstall lower rail wheels.


INSTALL TRANSVERSE RAIL

1. Raise one rail (item 9) and two rail spacers (item 14) up to the adapter carriages (item 1) and hold in place. Place a pair of rail clips (item 15) around the rail flanges, locating rail clips under the mounting holes in the adapter carriages. Bolt the rail to the carriages in the three outside holes of the carriage:

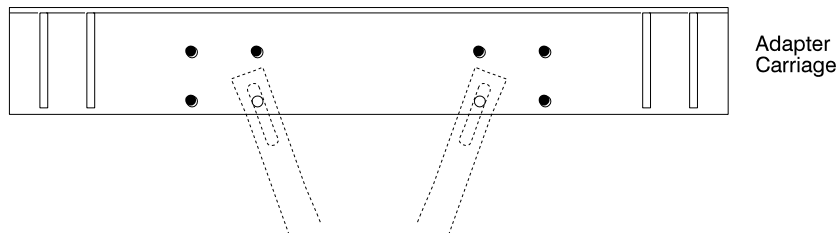

Lightly tighten hardware. Attach the second rail in the same manner.

2. Place the two triangulation bars (item 10) over the adapter carriages, locate the slots over the fourth mounting holes in the carriages, and lightly fasten bars in place.
3. Slide the rails back and forth to center them under the adapter carriages and to align both ends. Make adjustments at the triangulation bars and rail clips until the rails are spaced 17- 1/8" apart at both ends. When rails are in place, tighten hardware (16 places).

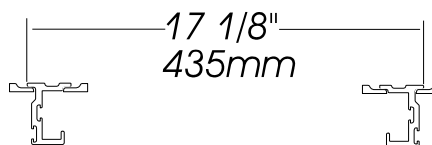

4. Adjust the lower, eccentric rail wheels on the adapter carriages for smooth travel.

INSTALL X/Y STOP & GUIDE WHEELS


1. Determine which rail faces the opposing system in the room, then slide the X/Y stop (item 20) down the outside groove of that transverse rail. Leave hardware loose; tighten when system is fully installed and the collision position is identified.
2. Slide the four guide wheel assemblies onto the transverse rails, one assembly at each rail end. Slide the four assemblies in until the wheels contact the OTC rails, then tighten the nut plate screws. Use **Loctite #242** on hardware.

INSTALL RAIL END STOP *(Refer to End Stop Assembly Drawing)*

NOTE: End plates have been provided for the transverse rails. Install end plates at time of system installation. See system instructions.

WARNING! Failure to install end plates will result in serious injury to user.

1. Slide the end stop assembly (item 1-6) down to the far end of one rail, making sure rubber bumper (item 5) faces in away from end plate. Do not tighten hardware. Adjust end stop positioning at time of system installation. See system instructions.

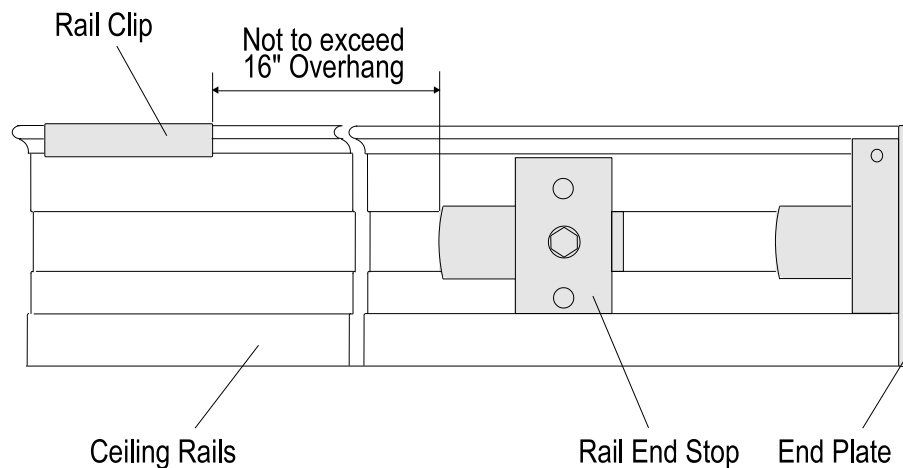

WARNING! The position of the Rail Stop must not exceed 16" rail overhang or damage to the system may result.

INSTALL RAIL END PLATE *(Refer to End Plate Assembly Drawing)*

1. Install one end plate (item 22) at the far end of the rails, making sure rubber bumper (item 23) faces in towards the rails. Use **LOCTITE #242** on all threads.


END-LOAD GANTRY *(Refer to End Plate Assembly Drawing)*

1. Remove shipping straps and packing material from carriage. Using Hi-Jack or other suitable lift table, raise gantry (still attached to pallet) up and into the rails. Adjust center wheels if necessary per figure on page 8. ***Make sure all carriage wheels are engaged onto rails before lowering Hi-Jack.*** Remove pallet and shipping bars from support unit.
2. Install the second end plate (item 22) at the near end of the rails, making sure rubber bumper faces in towards the carriage. Use **LOCTITE #242** on all threads.

WARNING! Failure to install both rail end plates
may result in serious injury to user.

INSTALL MONITOR MOUNT *(Refer to Monitor Mount Assembly Drawing)*

NOTE TO INSTALLER: It is important that the load be balanced on the monitor mount. Take into account any differences in size and weight of the monitors.

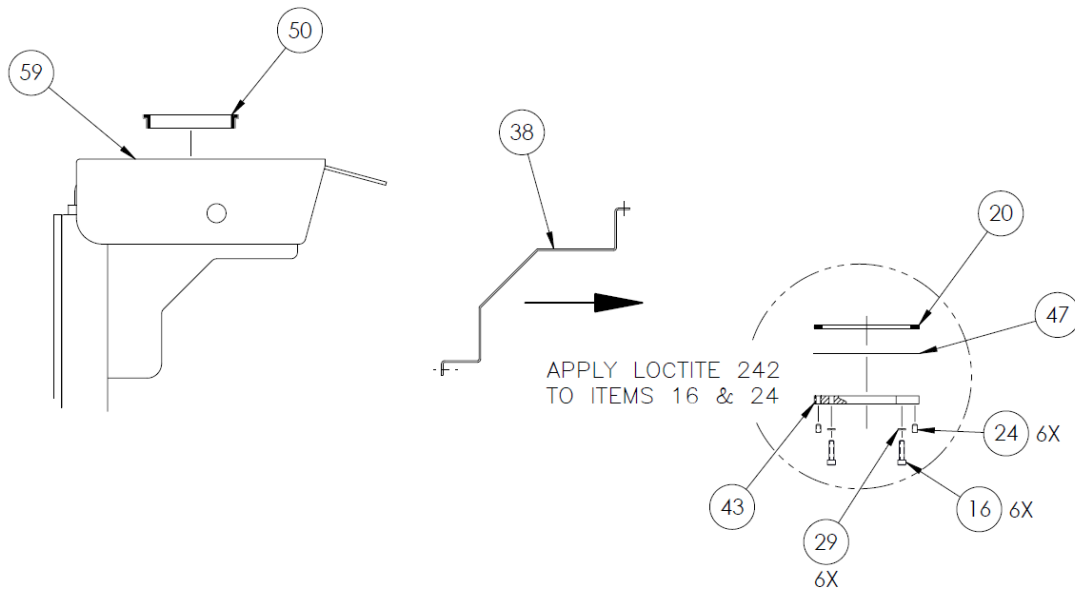


1. Remove wireway cover (item 38).
2. Drop rotation bearing (item 50) into pivot plate (item 59).

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use
with OTC Adapter


3. Take the thicker thrust ring (item 20) and put around rotation bearing (item 50).
4. Place the stainless-steel thrust ring (item 47) below the thicker thrust ring (item 20).
5. The six bolts (item 16) with lockwashers (item 29) go through rotation cap (item 43) and rotation bearing (item 50) into the pivot tube (item 43 in the Core Assembly Drawing). Use **LOCTITE #242** on hardware.
6. Apply a very little amount of **LOCTITE #242** to and insert the six setscrews (item 24) into the six threaded holes on second rotation cap (item 43). Thread the setscrews in until they contact the rotation cap (item 43). The setscrews may be tightened or loosened later to adjust rotational resistance.

WARNING! Failure to securely attach hardware may result in serious injury to patient, user, and damage to equipment.

INSTALL MONITOR (*Refer to Monitor Mount Assembly Drawing*)

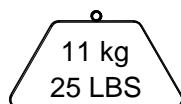

Maximum weight of any individual monitor.

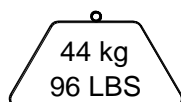

Maximum total weight of monitors.

1. To attach a monitor to the system, first remove a monitor mounting bracket (item #30) by removing its mounting screw (item #21). Fasten the bracket to the monitor (hardware not provided). Reinstall the bracket with the monitor.

WARNING! Failure to securely attach hardware may result in serious injury to patient, user, and damage to equipment.

2. Monitors can be tilted and rotated.
3. If monitor tilt resistance is inadequate and it fails to stay in the desired position, tighten bolt (item 21).

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use
with OTC Adapter


4. If monitor rotation resistance is inadequate and it fails to stay in the desired position, tighten bolt (item 22).

NOTE TO INSTALLER: Ground stud (item 27) has been provided to properly ground monitors as a group. Terminal strip (item 31) has been provided for distribution of line voltage to each monitor. Cut and strip each monitor power cord to fit.

WARNING! Remove plastic cover and remove block of wood between gantry core and arm. Failure to do so may result in serious injury to user and damage to equipment.

SET ROTATIONAL LIMITS (*Refer to Monitor Mount Assembly Drawing*)

If desired, the stop pin (item 4) can be relocated to another of its four possible mounting locations. Remove rotational cover (item 35) to access the stop pin mounting screw (item 52).

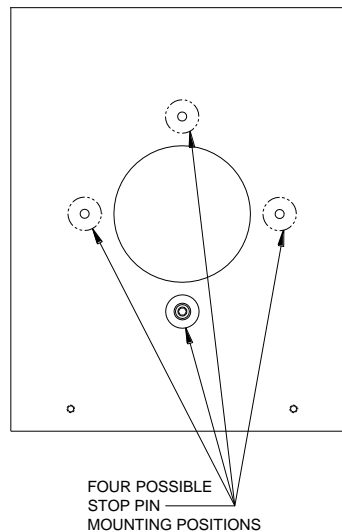

ELECTRICAL CONNECTIONS


1. Colors of conductors in power supply cords shall be in accordance with IEC publication 60227 (amendment #1) or with IEC publication 60245.

Wiring Color Codes:

International: brown (line), blue (neutral), green/yellow (ground)

North America: black (line), white (neutral), green/yellow (ground)

2. Cable tie mounting platforms are supplied near the terminal block for anchoring the monitor cords and the wires from the mains. Use nylon cable ties. (Tying the cord into a knot or tying the ends with string shall not be used for cord anchorage).
3. Conductors of the power supply cord shall be arranged that the protective earth conductor is not subject to strain as long as the phase conductors are in contact with their terminals.
4. All power cords used for mains connection shall have double insulation. Conduit provided for routing cables shall not be relied on for insulation.

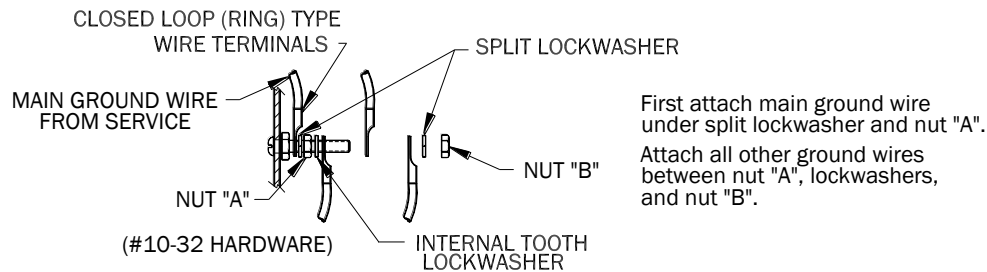

5. Connect all monitor ground wires and the service ground wire as shown in the figure above. (The number of wires varies with the number of monitors installed).


INSTALL STATIONARY CABLE HANDLING ASSEMBLY

(Refer to Stationary Cable Handling Kit Drawing)

1. Determine which rail end the *stationary* cable handling kit will be positioned. If, at the far end of the rail, slide kit into rail track; slide kit into rail track *prior* to positioning the *mobile* cable handling kit. If stationary kit is to be positioned at the near end of rail, slide into rail *after* positioning mobile cable handling kit. Stationary kit is typically mounted approximately 6 inches (15cm) from rail end.
2. After applying LOCTITE #242 to threads, tighten the setscrew (item 4) to secure in place.
3. Remove retainer (item 2) by taking out the two mounting screws (item 3).

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use
with OTC Adapter

4. Place section of cable hose into clamp (item 1).
5. Reinstall retainer (item 2) screws (item 3).

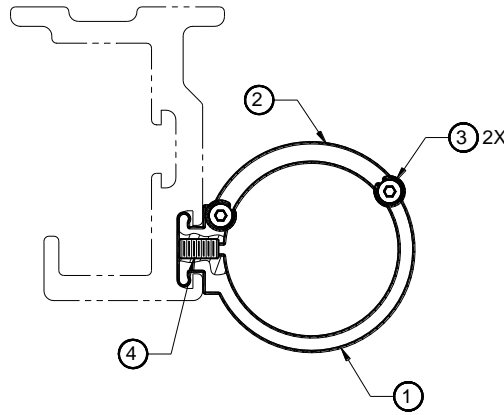


STATIONARY CABLE HANDLING KIT ASSEMBLY

INSTALL MOBILE CABLE HANDLING ASSEMBLY (*Refer to Cable Handling Kit Assembly Drawing*)

1. Slide nut plates (item 4) of two hanger standoff and safety bracket assemblies into rail track. After **applying LOCTITE #242** to the end that will be secured into the nutplate, position standoffs approximately 48" (122cm) from each other. To assure proper fit and positioning, temporarily assemble cable hanger shaft (item 2) to standoffs. Place one shaft support (item 1) onto far end standoff. Place the other shaft support to the second standoff, while temporarily installing cable hanger shaft (item 2) into holes of shaft supports. Lightly tighten socket head cap screws (item 3). **Use LOCTITE #242 on hardware.**

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use
with OTC Adapter

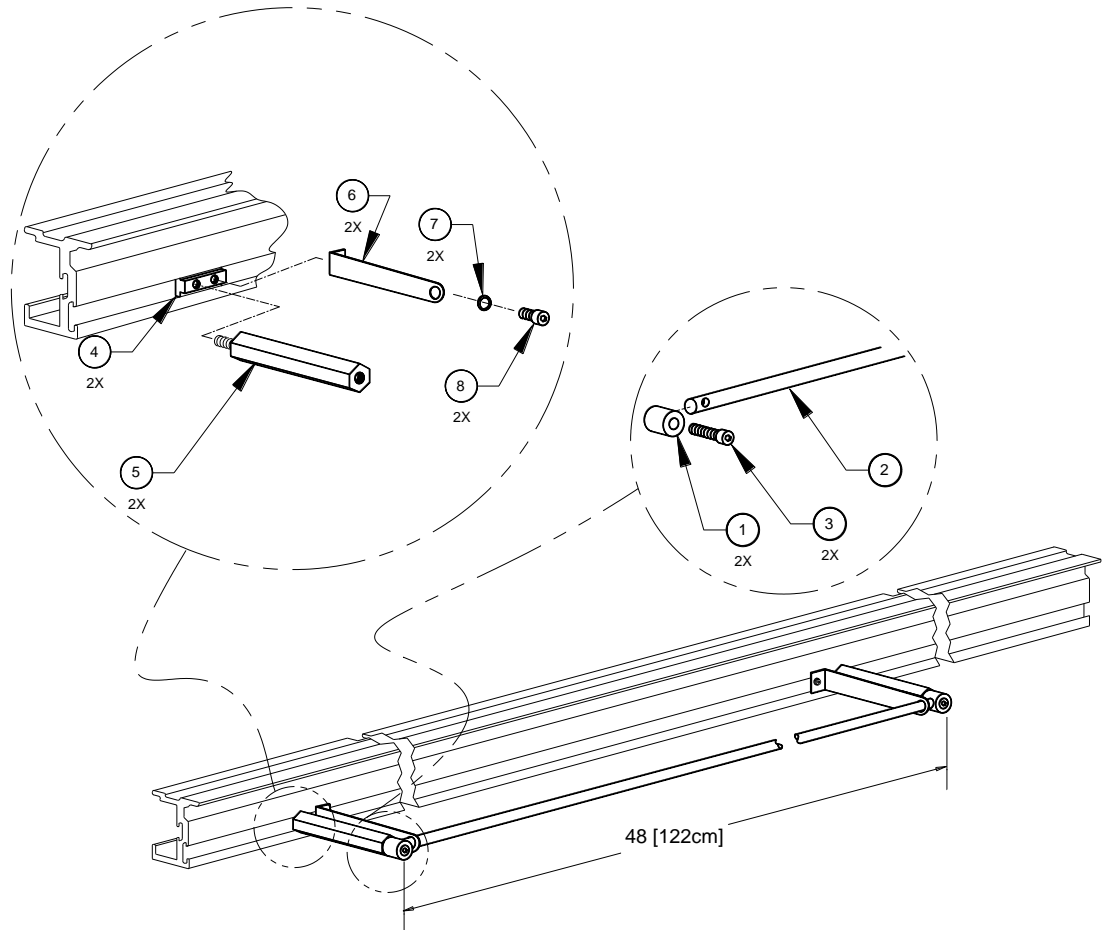


MOBILE CABLE HANDLING KIT ASSEMBLY

2. Position shaft/cable handling subassemblies (items 1-8) to desired location on rail. Without disrupting alignment, remove shaft and shaft supports. Tighten safety bracket (item 6) screws (item 8). Torque standoffs to ~35-50 inch pounds, 4.2-6.0(Nm). **Use LOCTITE #242 on hardware.**

CAUTION! Over tightening the standoff will weaken or break the set screw connecting the standoff to the nut plate causing attached components to fall. **DO NOT** loosen or adjust the exposed length of the setscrew on the standoff.


3. Reattach shaft supports. Install one end of shaft to a shaft support. Tighten socket head cap screw (item 5). Slide cable trolley assembly(s) onto cable hanger shaft (item 4). Install free end of shaft into other shaft support. Tighten second socket head cap screw (item 4). **Use LOCTITE #242 on hardware.**

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use with OTC Adapter

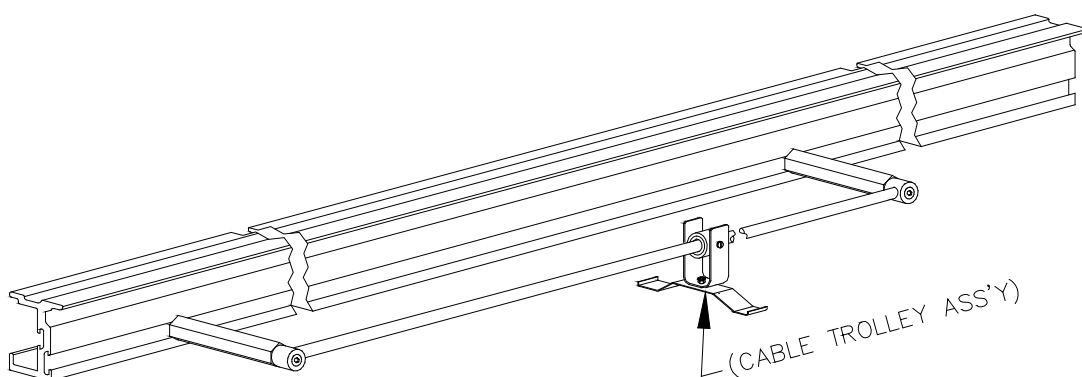

DRESS CABLES *(Refer to Gantry Core Assembly)*

1. When the monitors have been properly installed, it is recommended that the cables be brought through the pivot tube (item 43). 2" (5cm) flexaust (provided) is to be attached to pivot tube and to be routed underneath the arm (item 51) via the cable clamps (items 46, 50). Rotate entire system and bring to maximum positions: the system should not drift from any position. If it does, check that enough cable has been left to allow freedom of motion.

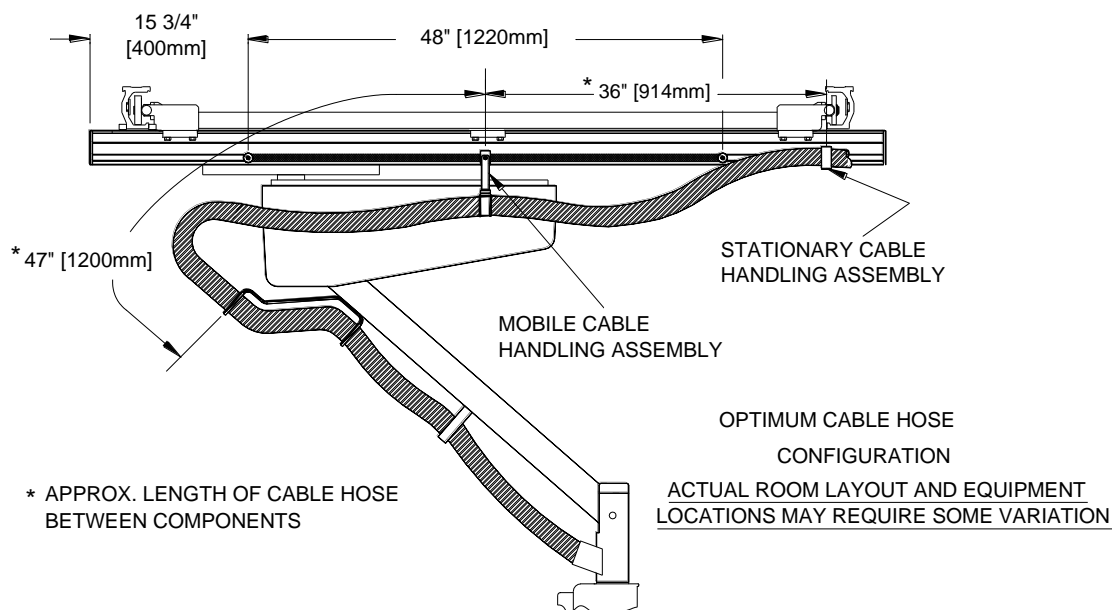

BALANCE SYSTEM (Refer to Gantry Core Assembly)

Note: System balance has been set at factory based upon quoted monitor weight.


1. Note initial settings from Spring Data label (item 18). Substantial increases in payload may require spring change.
2. *Prior to adjustment* - payload must be secure and arm *horizontal*. Fine adjustment to compensate monitor counterbalance can be made by turning adjustment screw (item 53). Rotate adjustment nut:



clockwise to increase capacity



counter clockwise to decrease capacity

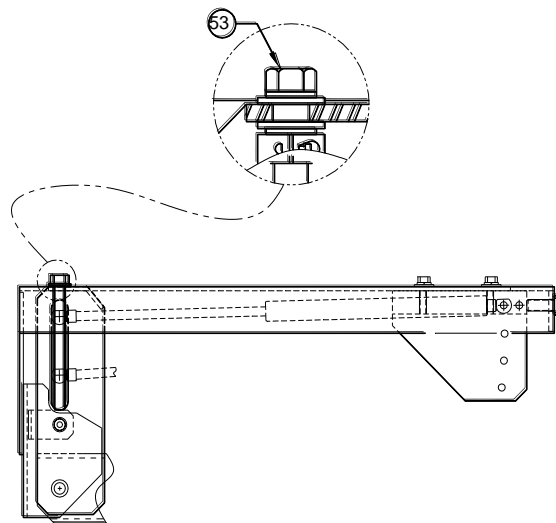

Yoke Balance Adjustment

WARNING! Yoke has been leveled at manufacturers. Make adjustment only if absolutely necessary to compensate for an unbalanced load.

NOTE: *Prior to yoke adjustment* - payload must be secure and arm *horizontal*.

1. Fine adjustment, to compensate for yoke balance, can be made by turning the link (item 48). Loosen threaded hex nut (item 24) at outboard rod end (item 10); spin link until yoke is in a horizontal position; retighten hex nut.

MONITOR MOUNT ASSEMBLY ROTATION RESISTANCE

1. The setscrews (item 24) installed in step 6 of "INSTALL MONITOR MOUNT" on page 12 can be loosened or tightened to adjust the rotation resistance of the suspended monitor mount assembly.
2. Replace wireway cover (item 38) after making adjustment.

F. CLEANING THE EQUIPMENT

No part of this unit is designed to be sterilized in an autoclave. Do not allow water or other liquids to enter the equipment as this may cause short circuits or corrosion. Clean parts with a clean cloth dampened with disinfectant or a mild detergent solution. Do not use abrasives, solvents, sprays or corrosive cleaning agents. Gently rub with a clean soft cloth to dry.

If room is to be disinfected by means of an atomizer, the equipment must be covered with plastic or similar sheeting. The equipment must be turned off well in advance of this procedure to prevent convection currents from drawing the disinfectant mist into the equipment. After the mist disperses completely, the sheeting may be removed and the equipment disinfected as described above.

G. MAINTENANCE

"Authorized Technician"

All maintenance procedures should be done by an experienced technician with demonstrated knowledge and skills (electrical and mechanical) relative to this type of equipment.

This individual must have access to this manual and the proper tools.

Daily Maintenance Checks:

1. Pre-Operational and Post-Operational Checks


Perform daily checks of the monitor suspension BEFORE and AFTER operating the equipment.

CAUTION! If any abnormality is found in the monitor suspension, stop using it. Post a sign reading "DO NOT USE" so that the system is not used by mistake. Then contact your Toshiba representative for inspection and repair.

2. Visual check


Before checking the operation, confirm the following:

- The monitor suspension is not tilted. Refer to the following figures.
- Although "A" in the following figure shows a gap, there is no gap at the monitor frame rotation section.
- Monitors installed on the monitor suspension are not tilted.
- Screws or some parts are not loosened or removed.
- The pad mounted to the bottom of the monitor mount assembly is not damaged or missing.

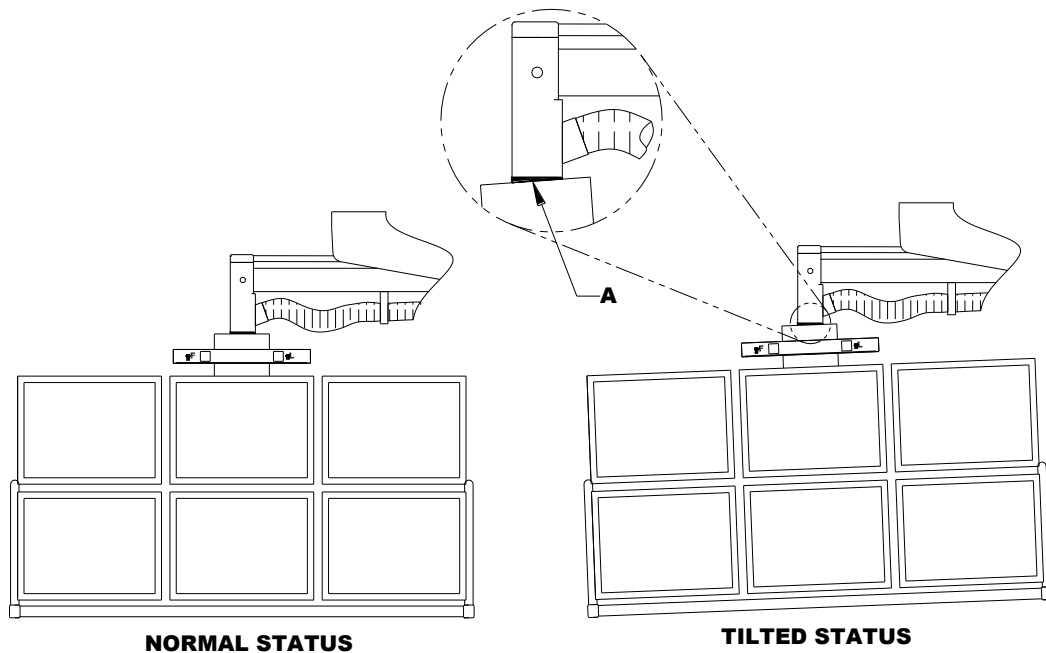

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use with OTC Adapter

Pad attached to the underside of the monitor mount bracket must be complete and undamaged

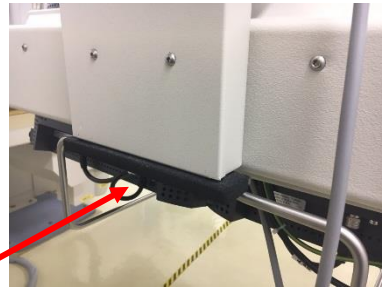


3. Operational check

Move all the movable sections and confirm the following:

- Operation does not lack smoothness compared to when the monitor suspension was installed.
- There is no play.
- There are no abnormal sounds.
- The monitors or monitor suspension are not tilted due to looseness of secured sections.


Annual Maintenance Check:

1. Clean carriage wheels and guides and the rails where carriage wheels and guides ride inside the rails.

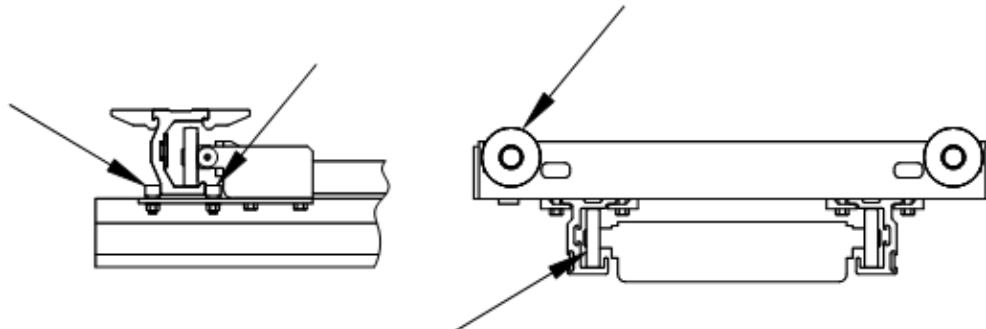


Installation and Maintenance Guide for IDI 1000F Series Suspension System for use
with OTC Adapter

2. Check that the rail mounting hardware has not become loose.

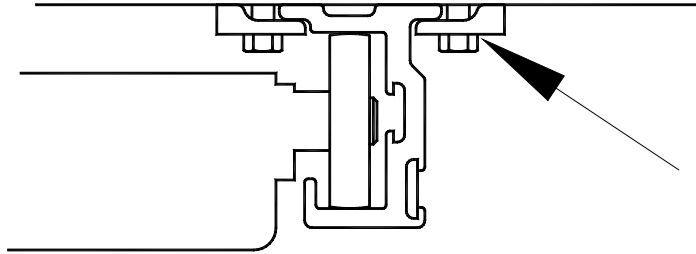


3. Check that the hardware securing the end plates are tight.

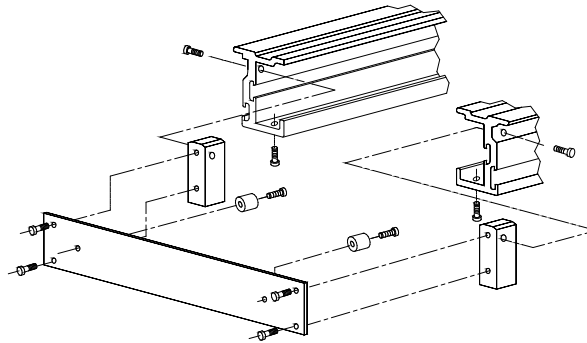

4. Perform a complete functional inspection:
 - Rotate monitor mount assembly through complete range.
 - Raise and lower monitor mount assembly.
 - Roll carriages through complete range of rails.
5. Clean and touch up the painted surfaces.
6. Check that the hardware is tight and that there is no wear at (A) the end of the gantry core arm, (B) the rotational section between the gantry core arm and the monitor mount assembly, (C) the monitor mount assembly pivot joint, (D) the monitor bracket pivot (E), and the monitor. Should the screws labeled (F) become loose, there will be an opening of the mechanical junction between the gantry core arm and the monitor mount assembly (G).

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use
with OTC Adapter

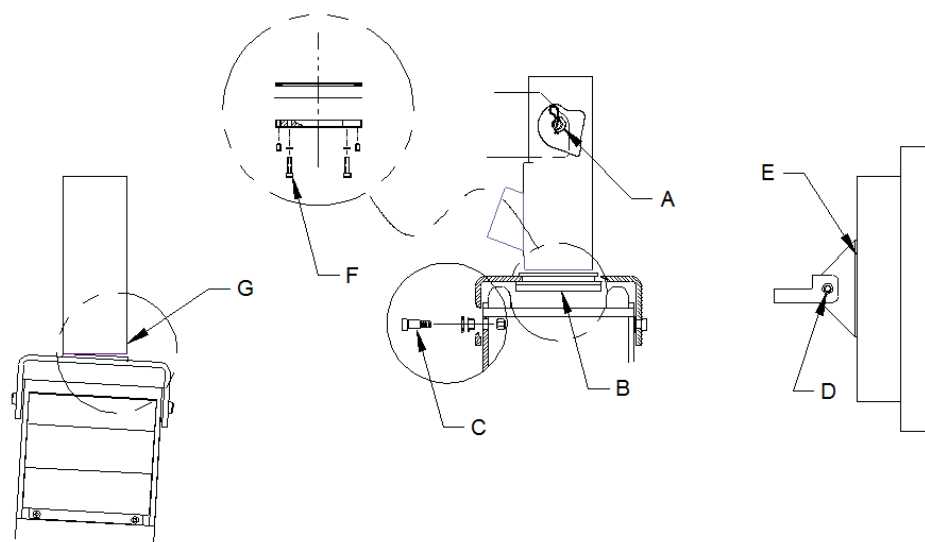

7. Check the pad attached to the underside of the monitor mount. If there is any damage to the pad or if the pad is not properly attached to the unit, it must be replaced.



The procedures mentioned in these instructions are based on the recommendations of HHS, I.E.C., etc.

In cases of increased wear or where severe working conditions exist, maintenance checks should be made at shorter than the specified intervals.

Should any control or indicator fail to operate properly, do not use the equipment until a repair has been affected. Operating equipment with defective components may expose the operator or the patient to safety hazards.

The manufacturer strongly recommends that a maintenance program be initiated and that a maintenance record be kept detailing dates and nature of maintenance performed, the name of the service engineer, and any other relevant information.

WARNING! Removal of any hardware requires re-assembly with Loctite #242 or equivalent (depending on hardware size) following manufacturer's instructions.

H. CUSTOMER SUPPORT

For technical assistance, be sure you have the complete model and serial number before contacting the local or national service office.

I. DISPOSAL GUIDELINES



Most of the components used in this suspension system are made of metal, usually aluminum or steel, and are easily recycled.

The gas springs are nearly 100% recyclable. They must be decompressed, with the remaining oil drained, before processing. (The compressed gas inside is nitrogen).

J. APPROVALS



SGS Testing Service



The system was tested and found to be in compliance with the requirements of all relevant directives and standards in effect within the European Union at the time of manufacture.



European
Authorized
Representative:

Advena Ltd. Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

REVISIONS TO THIS MANUAL

<u>Rev</u>	<u>Date</u>	<u>Description</u>
A	March 2004	Release
B	May 2004	Rev 2 BOM Changes
C	August 2008	ECO 294
D	September 2011	ECO 319

Installation and Maintenance Guide for IDI 1000F Series Suspension System for use with OTC Adapter

E

May 2018

ECO E571 Added warnings for pad