



ITEM NO.	PART NUMBER	DESCRIPTION	QTY
1	A240-1140	KIT, 56IN. MONITOR MOUNTING HARDWARE	1
2	C000-0207	TAPE,FACED,DBL,.005x2 " WIDE	21IN.
3	C000-0706	BUMPER, URETHANE, W/INSERT, 3/4" DIA X 3/4" LG	1
4	C000-0929	Plug, Finishing, 1" Tube OD, .84-.94 ID, Black	6
5	C000-1618	PLUG,PLASTIC,.71-.8IN SQ HOLE	4
6	J000-0007	BLOCK, TERMINAL, 12POS.	1
7	J000-0056	TIE WRAP ANCHOR	4
8	P240-1125	MONITOR MOUNT,WELDMNT,HD	1
9	P240-1127	MOUNT, HANDLE, LD SUSPENSION	6
10	P240-1129	COVER, BACK, MONITOR MNT, LD SUSPENSION, PAINT	1
11	P240-1130	BRACKET,MONITOR HANGER,HD	2
12	P240-1131	COVER,PIVOT, MONITOR MOUNT, LD SUSPENSION	1
13	P240-1132	BRACKET, TIE DOWN BAR, LD SUSPENSION	2
14	P240-1141	PLATE, METER BKT	1
15	Q240-1045	RING, THRUST (MO) 3.386 X 4.063 X .125	1
16	Q240-1049	CAP, ROTATION,1000F	1
17	X240-1143	PAD, MONITOR HANGER	1
18	Z240-1047	BUSHING, PIVOT, 1000F	1
19	Z240-1115	TUBE, HANDLE, 1.00 X .065 WALL	3
20	Z240-1122	PIN, LOCKING, LD SUSPENSION	4
21	Z240-1134	PAD, HANDLE, LD SUSPENSION	4
22	Z240-1136	RING, THRUST, SS, 1000F-1H	1
23	Z240-1139	PAD, HANDLE, SHORT, LD SUSPENSION	2
24	FLOOR STOCK	WSHR,FLT,ZP,#10 SAE	10
25	FLOOR STOCK	WSHR, LOCK, TOOTH, ZP, #10	1
26	FLOOR STOCK	WSHR,LCK,SPL,ZP,#10	9
27	FLOOR STOCK	SSCR,SOC,CUP,ALY,#10-32 X 5/16	6
28	FLOOR STOCK	SL-RHMS 0.19-32x1x1-N	1
29	FLOOR STOCK	SCR,SHCS,SST,#10-32 X 1	6
30	FLOOR STOCK	SCR,SHCS,ALY,#10-32 X 3/4	7
31	FLOOR STOCK	SCR,SHCS,ALY,#6-32 X 5/8	3
32	FLOOR STOCK	SCR,SOC,BTN,SST,1/4-20 X 1 1/2	12
33	FLOOR STOCK	SCR,SOC,BTN,SST,#10-32X2-1/2	4
34	FLOOR STOCK	SCR,SOC,BTN,SST,#10-32 X 1/2	2
35	FLOOR STOCK	SCR,SOC,BTN,SST,#10-32 X 3/8	8
36	FLOOR STOCK	SCR,PHH,PAN,ZP,#8-32 X 1/4	4
37	FLOOR STOCK	NUT,HEX,ZP,#10-32	6
38	FLOOR STOCK	NUT,HEX,K-TYPE,ZP,#10-32	1
39	FLOOR STOCK	NUT,HEX,K-TYPE,ZP,#10-32	2

NOTES:
1. FOR QUANTITIES AND LOCATION OF LABELING, REFERENCE IDI DRAWING NUMBER A240-1402.

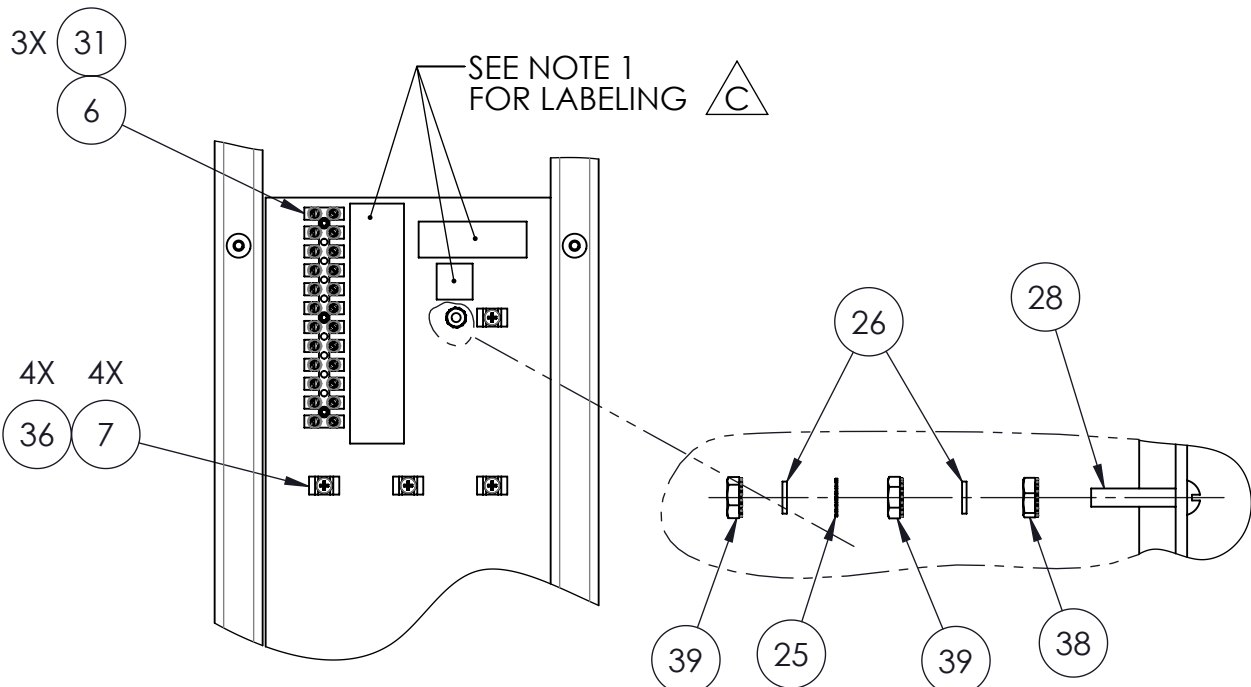

VIEW OF CONNECTION PANEL
FROM BACKWITHOUT COVER

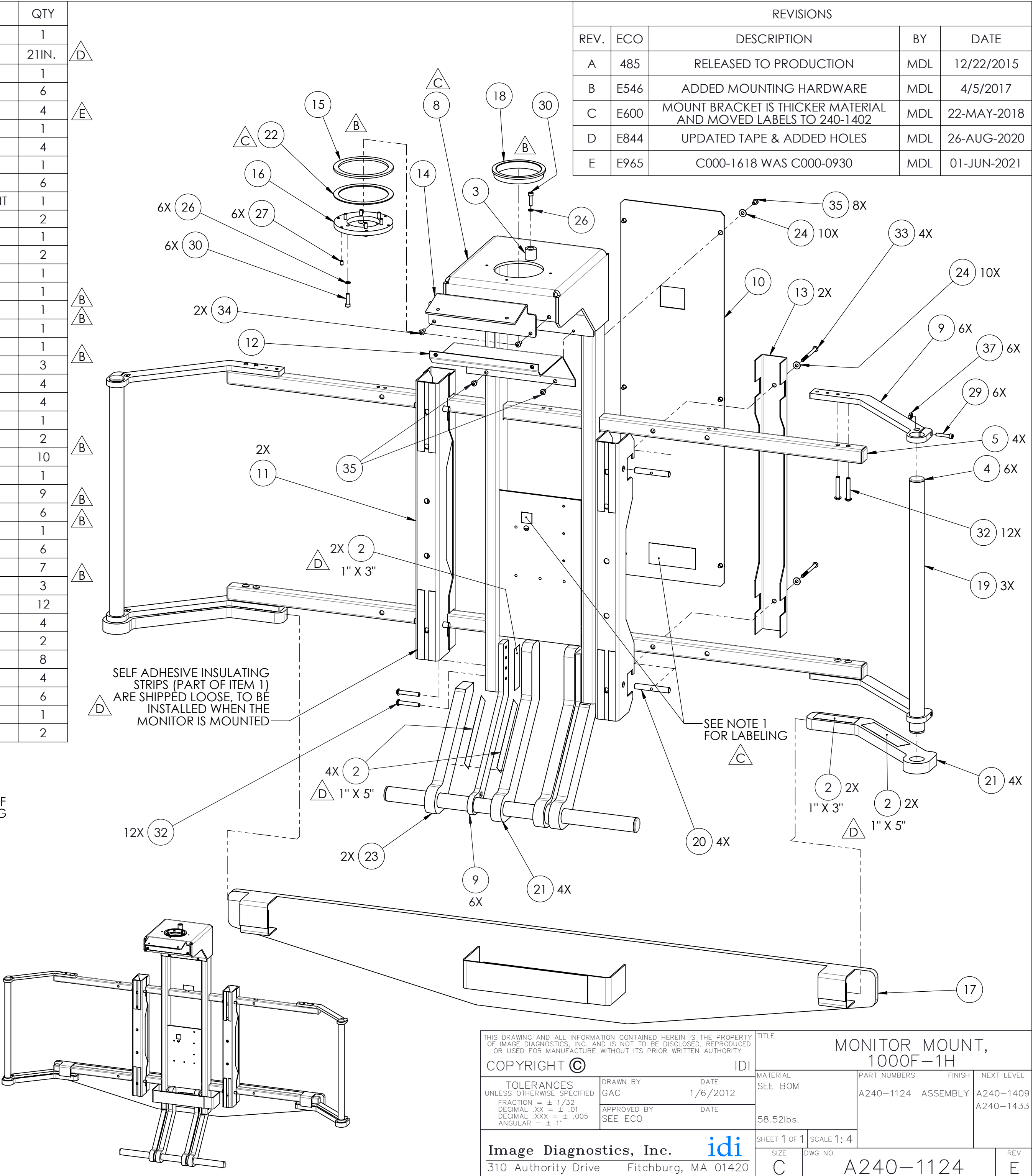

REVISIONS				
REV.	ECO	DESCRIPTION	BY	DATE
A	485	RELEASED TO PRODUCTION	MDL	12/22/2015
B	E546	ADDED MOUNTING HARDWARE	MDL	4/5/2017
C	E600	MOUNT BRACKET IS THICKER MATERIAL AND MOVED LABELS TO 240-1402	MDL	22-MAY-2018
D	E844	UPDATED TAPE & ADDED HOLES	MDL	26-AUG-2020
E	E965	C000-1618 WAS C000-0930	MDL	01-JUN-2021

THIS DRAWING AND ALL INFORMATION CONTAINED HEREIN IS THE PROPERTY OF IMAGE DIAGNOSTICS, INC. AND IS NOT TO BE DISCLOSED, REPRODUCED OR USED FOR MANUFACTURE WITHOUT ITS PRIOR WRITTEN AUTHORITY				
COPYRIGHT ©		IDI		
TOLERANCES UNLESS OTHERWISE SPECIFIED FRACTION = ± 1/32 DECIMAL .XX = ± .01 DECIMAL .XXX = ± .005 ANGULAR = ± 1°		DRAWN BY GAC	DATE 1/6/2012	MATERIAL SEE BOM
APPROVED BY SEE ECO		DATE	58.52lbs.	PART NUMBERS A240-1124
Image Diagnostics, Inc. idi		310 Authority Drive	Fitchburg, MA 01420	FINISH ASSEMBLY
SHEET 1 OF 1		SCALE 1:4	NEXT LEVEL A240-1409 A240-1433	
SIZE C		DWG NO. A240-1124		REV E