



ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	C000-0044	BUMPER, RUBBER, BLK 1x1 RND	1
2	C000-0050	BRG,RL,1 BORE	1
3	C000-0054	BRG,CUP,2.24 O.D. USE W/C000-0050	1
4	C000-0057	COLLAR, CLAMP 3/8-24	1
5	C000-0066	BRG,FL,1/2 X 3/4 X 1	2
6	C000-0068	BRG,TH,3/8 X 3/4 X 1/32	2
7	C000-0069	BRG,TH,1/2 X 1 X 1/16 BOS TB-816	1
8	C000-0070	ROD END 1/4-28 LEFT HAND	1
9	C000-0071	ROD END 1/4-28 RIGHT HAND	1
10	C000-0179	WSHR,FLT,NYL,3/16 X 3/8 X 1/32	4
11	C000-0212	SCR,SOC,SHLDR,SST,1/4 X 3/4	1
12	C000-0286	BRG,TH,1/2 X 1 x 3/32	1
13	C000-0352	BRG,TH,1 3/4 X 2 3/4 X 1/16	1
14	C000-0353	BRG,PL,1 3/4 X 2 X 1 1/2	1
15	C000-0541	STNDF,HEX,#8-32x5/8x3/8	4
16	C000-0586	SPRING,GAS,1300N W/STUD,CLIP&BOOT	2
17	C000-0194	SPRING,GAS,450N ON W/STUD & CLIP	2
18	C000-0679	WSHR,FLT,SST,13/32x7/8x.04-.06	2
19	C000-0928	BRG,FL,7/16 X 9/16 X 3/4	2
20	C000-0972	BUMPER, URETHANE W/INSERT 1x1x5/16	1
21	C000-1346	SHAFTING,SST,1/4 X 2-1/4	1
22	C000-1347	COLLAR,INSERT,BUMPER	1
23	P240-775	ADJUSTER, 1000F (PT)	1
24	P240-777	TUBE, PIVOT (PT)	1
25	P240-781	CORE, 1000F (PT)	1
26	P240-787	CAP, PIVOT, JOINT (PT)	1
27	P240-789	CLAMP, CABLE (PT)	1
28	P240-791	SLIDER,CLAMP,CABLE, 1000F (PT)	1
29	P240-834	COVER, SUSPENSION (PT)	1
30	P240-840	HANGER,MNT,HOSE,1000F (PT)	1
31	P240-1041	ROTATIONAL STOP 1000F (PT)	1
32	P240-1085	ARM 1000F-H1 (PT)	1
33	P240-1128	LINK,30.06",1000F-H1 CORE (PT)	1
34	Q240-1082	PIN, CROSS 3/4 DIA (EN)	1
35	X240-842	BUMPER 1/2x1x2-1/2 ..BLACK NEOPRENE	1
36	Z240-841	MOUNT, BUMPER	1
37	Z240-883	ADJUSTER .38 X 5 1/4 SS	1
38	Z240-1079	SSCR,SOC,CUST,SS 3/4-16x1.50	2
39	FLOOR STOCK	SCR,HEX HD,ZP,5/16-24 X 3/4 GR8	2
40	FLOOR STOCK	SSCR,SOC,CUP,ALLOY,1/2-13 X 2	1
41	FLOOR STOCK	NUT,CASTLE,ZP,1-14	1
42	FLOOR STOCK	NUT,HEX,ZP,1/4-28	1
43	FLOOR STOCK	NUT,HEX,JAM,ZP,1/2-13	1
44	FLOOR STOCK	NUT,NYLOK,3/8-16	1
45	FLOOR STOCK	PIN,ROLL,3/16 X 1 1/2	1
46	FLOOR STOCK	SCR,PHH,TRUSS,NP,#8-32 X 3/8	4
47	FLOOR STOCK	SCR,SOC,BTN,ALY,5/16-18 X 3/4	2
48	FLOOR STOCK	SCR,SOC,BTN,SST,#10-32 X 3/8	1
49	FLOOR STOCK	SCR,SHCS,ALY,#10-32 X 3/4	2
50	FLOOR STOCK	SCR,SHCS,ALY,1/4-28 X 1	1
51	FLOOR STOCK	SCR,SHCS,SST,#10-32 X 1	3
52	FLOOR STOCK	SCR,SOC,FLT,ALY,1/4-28 X 3/4	2
53	FLOOR STOCK	SCR,SOC,SHLDR,ALY,1/2 X 2-3/4	1
54	FLOOR STOCK	WSHR,LCK,SPL,ZP,5/16	3
55	FLOOR STOCK	WSHR,LCK,INT,ALY,#10	1
56	FLOOR STOCK	WSHR,FLT,ZP,5/16	2
57	FLOOR STOCK	WSHR,FLT,ZP,1 USS	1
58	FLOOR STOCK	SSCR,SOC,CUP,ALLOY,1/4-20 X 3/8.BRS	2
59	FLOOR STOCK	NUT,NYLOK,#10-32	1
60	FLOOR STOCK	SCR,SHCS,ALY,5/16-18 X 1	1
61	FLOOR STOCK	WSHR,FNDR,ZP,5/16 X 1-1/4	2
62	FLOOR STOCK	NUT,NYLOK,5/16-18	2
63	FLOOR STOCK	SCR,SOC,BTN,SST,#10-32 X 3/4	2

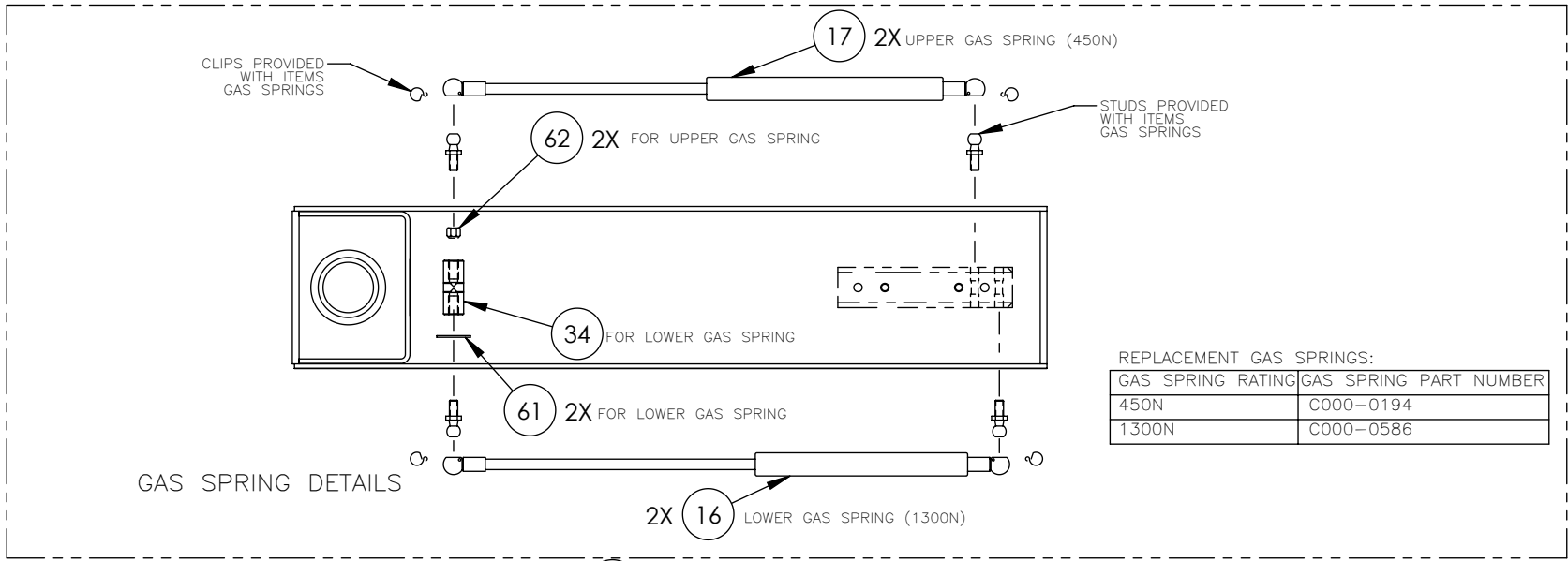

REVISIONS				
REV.	ECO	DESCRIPTION	BY	DATE
A	E909	RELEASED TO PRODUCTION	MDL	18-DEC-2020

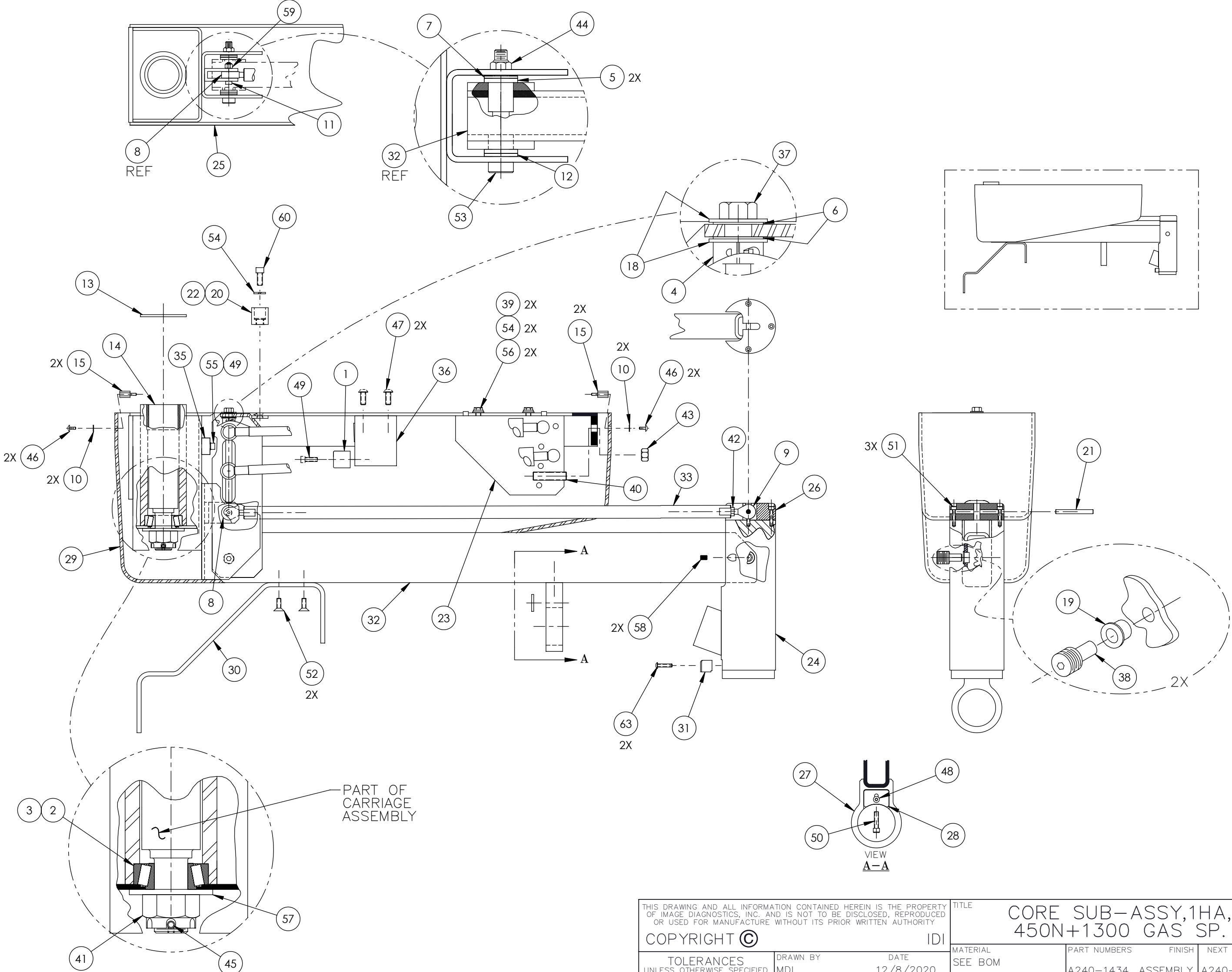

- NOTES:
- USE LOCTITE ON ALL THREADED FASTENERS.
 - FOR LOCATIONS OF LABELING, REFER TO IDI DRAWING A240-1435.

THIS DRAWING AND ALL INFORMATION CONTAINED HEREIN IS THE PROPERTY OF IMAGE DIAGNOSTICS, INC. AND IS NOT TO BE DISCLOSED, REPRODUCED OR USED FOR MANUFACTURE WITHOUT ITS PRIOR WRITTEN AUTHORITY.				TITLE CORE SUB-ASSY,1HA, 450N+1300 GAS SP.		
COPYRIGHT ©		IDI		MATERIAL SEE BOM	PART NUMBERS A240-1434 ASSEMBLY	FINISH A240-1433
TOLERANCES UNLESS OTHERWISE SPECIFIED FRACTION = ± 1/32 DECIMAL .XX = ± .01 DECIMAL .XXX = ± .005 ANGULAR = ± 1°		DRAWN BY MDL	DATE 12/8/2020			
		APPROVED BY SEE ECO	DATE			
Image Diagnostics, Inc. idi				SHEET 1 OF 1	SCALE 1:2	
310 Authority Drive Fitchburg, MA 01420				SIZE C	DWG NO. A240-1434	REV A