
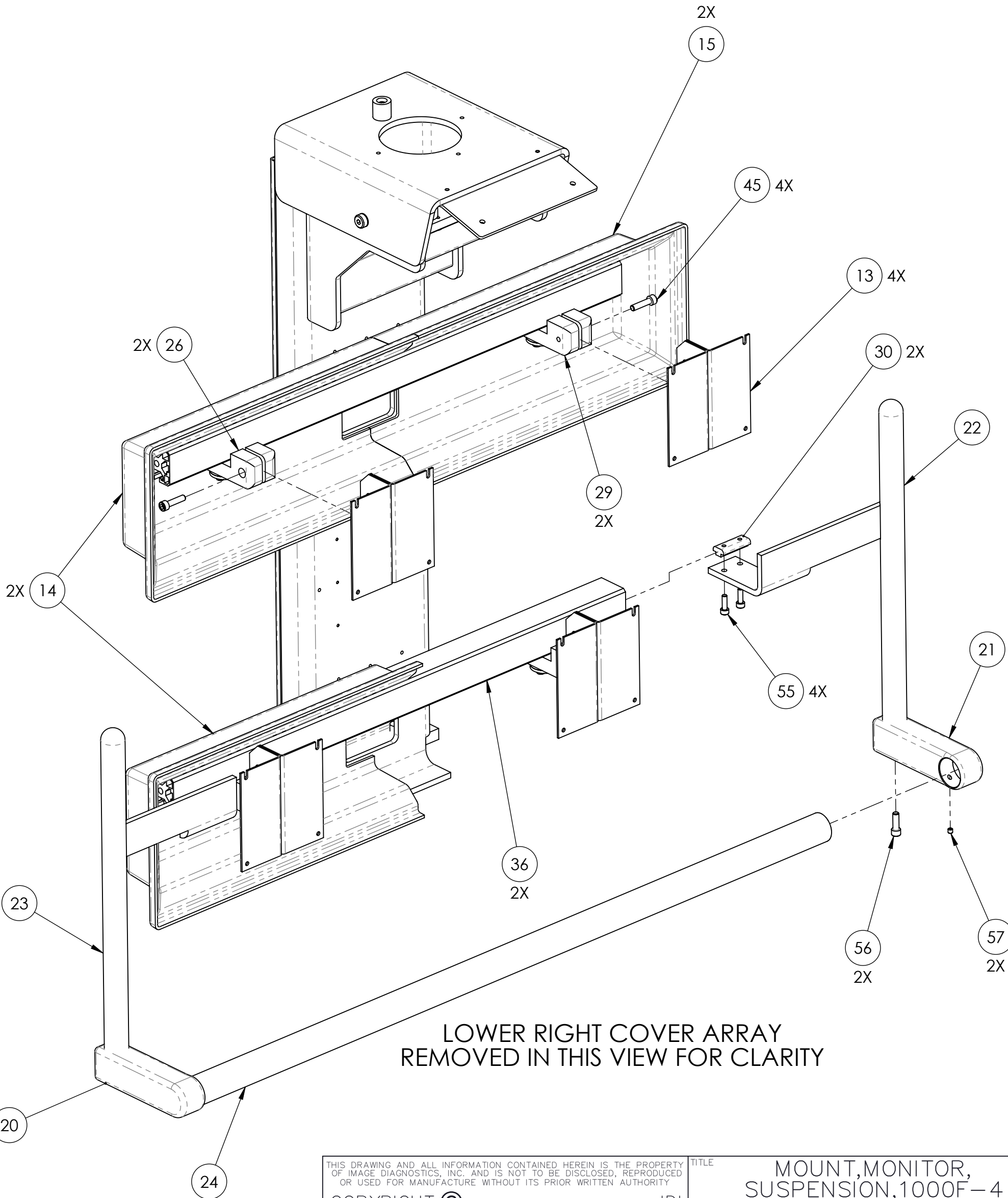
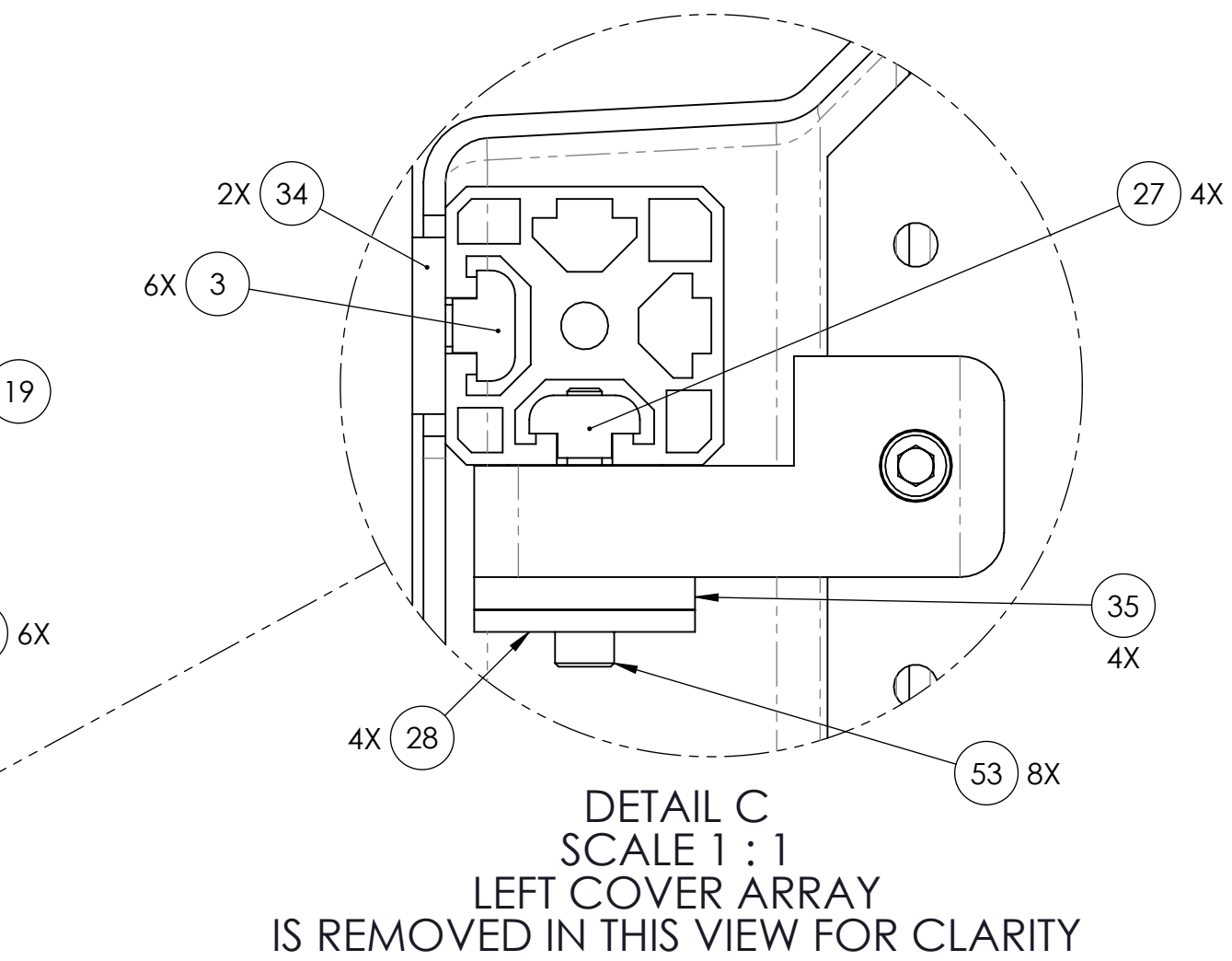
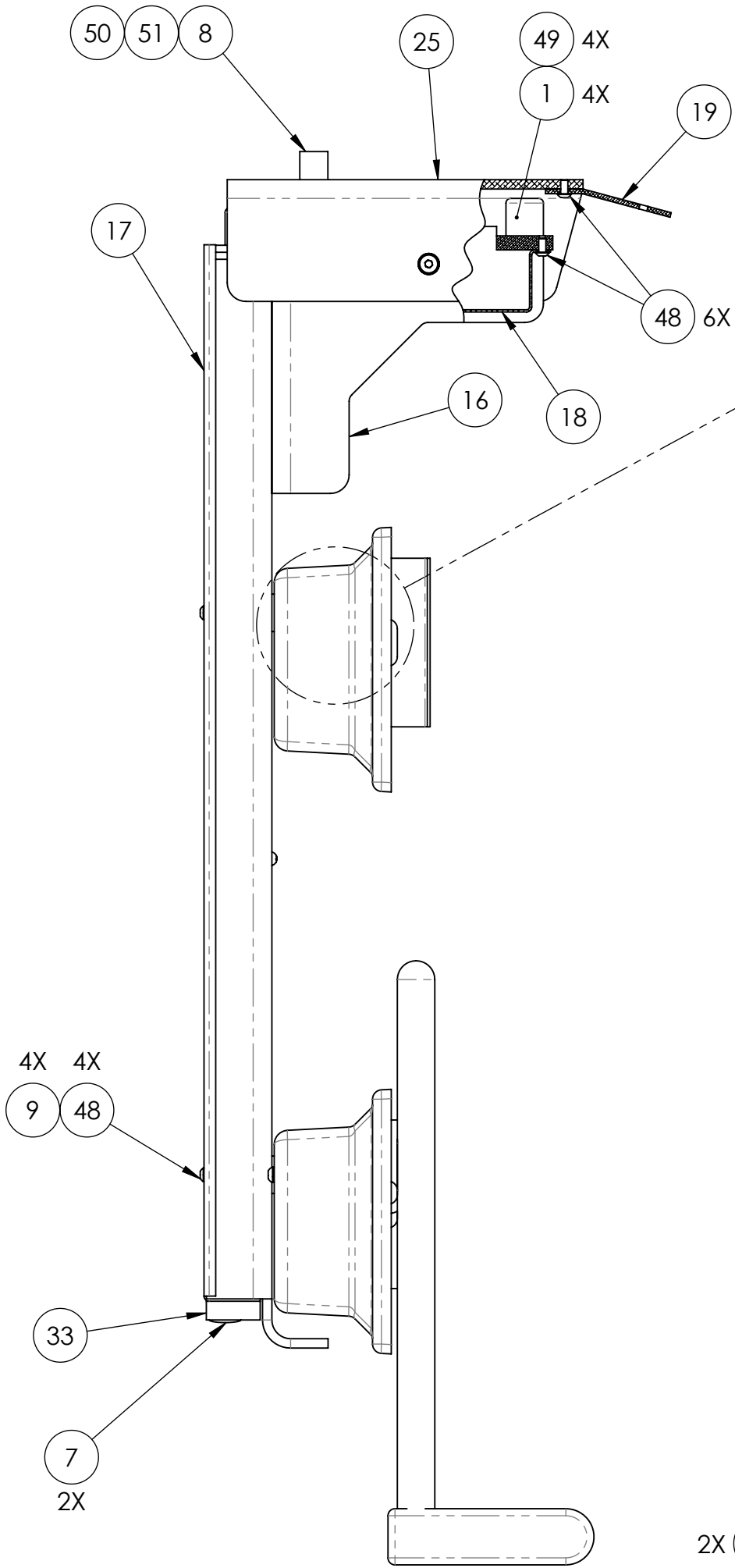
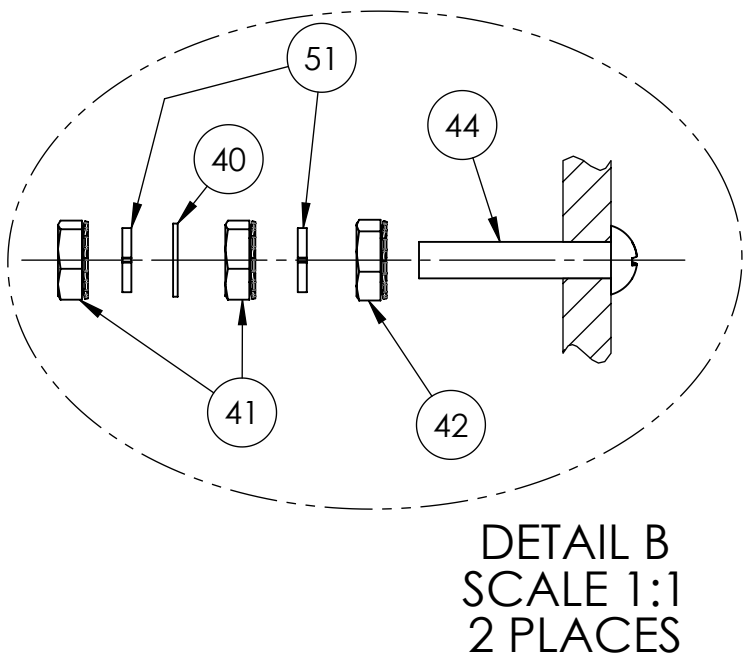
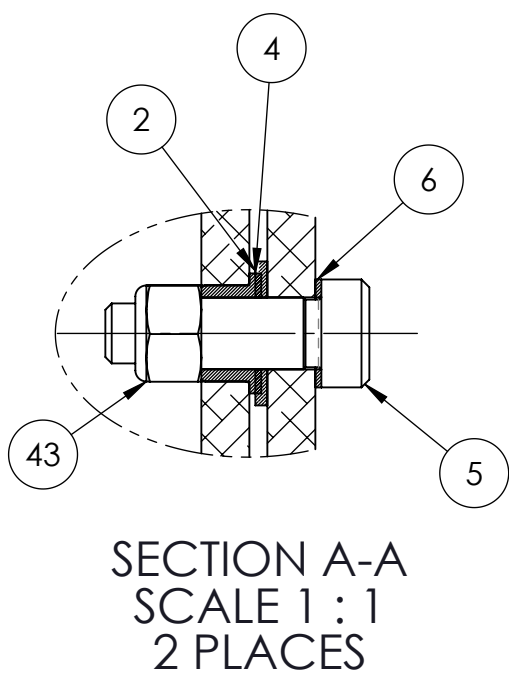


ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	C000-0044	BUMPER, RUBBER, BLACK - W/METAL INSERT, 1" DIA X 1" LG	4
2	C000-0046	BRG, TH, 3/8 X 3/4 X 1/16	2
3	C000-0533	NUT, T-SLOT, HD, M6	14
4	C000-0587	BRG, FL .375 X .50 X .3125	2
5	C000-0588	SCR, SOC, SHLDR, SST, 3/8 X 5/8	2
6	C000-0626	SPACER, SHAFT, .375 X .563 X .031	2
7	C000-0687	RIVET, PUSH, PLASTIC, BLACK	2
8	C000-0706	BUMPER, URETHANE, W/INSERT, 3/4" DIA X 3/4" LG	1
9	C000-1007	STNDF, M-F, 10-32 X 1-1/2 X 3/8 HEX	4
10	C000-1350	WSHR, FLT, NYLON, #12, .031x.25x.375	8
11	J000-0007	BLOCK, TERMINAL, 12POS.	2
12	J000-0056	TIE WRAP ANCHOR	12
13	P240-762	BRACKET, MONITOR, PAINTED	4
14	P240-793	COVER, ARRAY, LEFT, 14IN. (PT)	2
15	P240-795	COVER, ARRAY, RIGHT, 14IN. (PT)	2
16	P240-844	SUPPORT PLATE, BI-LEVEL SUSPENSIONS (PT)	1
17	P240-846	COVER, REAR, BI-LEVEL SUSPENSION (PT)	1
18	P240-850	COVER, WIREWAY, SUSPENSIONS (PT)	1
19	P240-860	PLATE, METER BRACKET, PAINTED	1
20	P240-865	CAP, HANDLE, LEFT (PT)	1
21	P240-867	CAP, HANDLE, RIGHT (PT)	1
22	P240-874	HANDLE, MOUNT, RIGHT, WELDMENT (PT)	1
23	P240-875	HANDLE, MOUNT, LEFT, WELDMENT (PT)	1
24	P240-889	HANDLE, 35IN., SUSPENSIONS (PT)	1
25	P240-914	PIVOT, TOP PLATE, MACHINED (PT)	1
26	Q240-818	PIVOT, MOUNT, MON, LEFT (CA)	2
27	Q240-873	NUT, T-SLOT, M5 X 1-1/4, EN	4
28	Q240-876	CAP, SWIVEL, SUSPENSIONS (EN)	4
29	Q240-893	PIVOT, MOUNT, MON, RIGHT (CA)	2
30	Q240-897	NUT, T-SLOT, M5 X 1-1/2, EN	2
31	Q240-1045	RING, THRUST (MO) 3.386 X 4.063 X .125	1
32	Q240-1049	CAP, ROTATION, (CA), 1000F	1
33	X240-1017	PAD, PROTECTIVE, SUSPENSION	1
34	Z240-814	SHIM, .188 X 1 X 5.625, SUSPENSIONS	2
35	Z240-851	BUSHING, SWIVEL, SUSPENSIONS	4
36	Z240-871	CROSSBAR, MONITOR MOUNT, 25IN.	2
37	Z240-1046	RING, THRUST, 3.386 X 4.063 X .020, 1000F	1
38	Z240-1047	BUSHING, PIVOT, 1000F	1
39	FLOOR STOCK	SCR, PHH, PAN, ZP, #8-32 X 1/4	12
40	FLOOR STOCK	WSHR, LCK, INT, ZP, #10	2
41	FLOOR STOCK	NUT, HEX, K-TYPE, BR, #10-32	4
42	FLOOR STOCK	NUT, HEX, K-TYPE, ZP, #10-32	2
43	FLOOR STOCK	NUT, NYLOK, 5/16-18	2
44	FLOOR STOCK	SCR, SLT, RND, BRS, #10-32 X 1	2
45	FLOOR STOCK	SCR, SHCS, SS, 6MM X 25MM	4
46	FLOOR STOCK	SCR, SHCS, SS, 6MM X 20MM	6
47	FLOOR STOCK	SCR, SHCS, ALY, #6-32 X 5/8	6
48	FLOOR STOCK	SCR, SOC, BTN, SST, #10-32 X 3/8	10
49	FLOOR STOCK	SCR, SHCS, ALY, #10-32 X 5/8	4
50	FLOOR STOCK	SCR, SHCS, ALY, #10-32 X 3/4	7
51	FLOOR STOCK	WSHR, LCK, SPL, ZP, #10	11
52	FLOOR STOCK	SSCR, SOC, CUP, ALY, #10-32 X 5/16	6
53	FLOOR STOCK	SCR, SHCS, SS, 5MM X 35MM	8
54	FLOOR STOCK	SCR, SOC, BTN, SST, M6 X 12MM	8
55	FLOOR STOCK	SCR, SHCS, SS, 5MM X 16MM	4
56	FLOOR STOCK	SCR, SHCS, SST, 1/4-20 X 3/4	2
57	FLOOR STOCK	SSCR, SOC, CUP, SS, #10-32 X 3/16	2

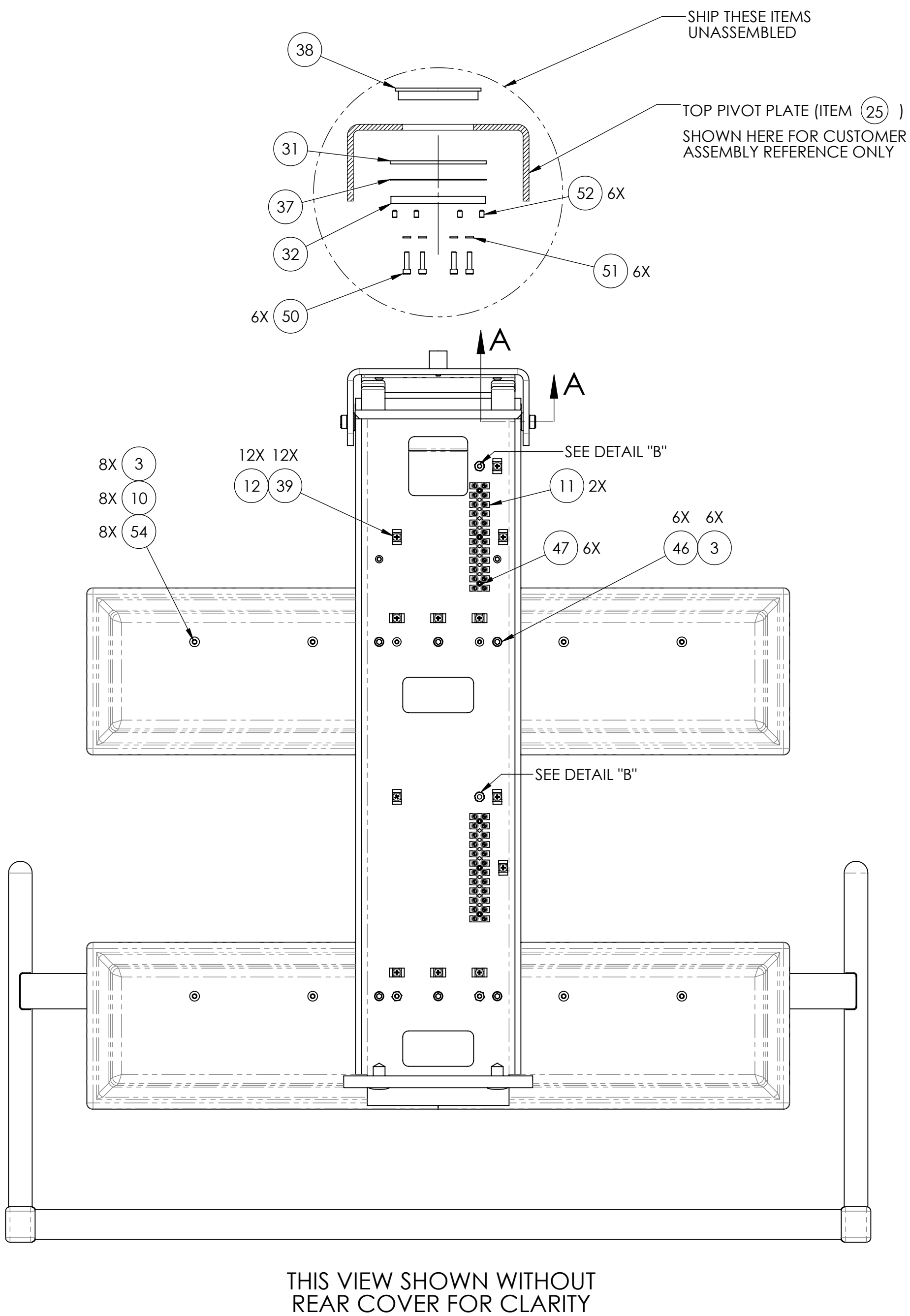

A240-932 NOTES:
1. APPLY LOCTITE TO ALL THREADED FASTENERS UNLESS OTHERWISE NOTED.
2. FOR LOCATIONS AND QUANTITIES OF LABELING, REFER TO IDI DRAWING #A240-1402.

REVISIONS				
REV.	ECO	DESCRIPTION	BY	DATE
A	425	ADDED C000-0046	MDL	19-NOV-2015
B	E669	ADDED BOM AND CONSOLIDATED LABELS	MDL	5-FEB-2019

THIS DRAWING AND ALL INFORMATION CONTAINED HEREIN IS THE PROPERTY OF IMAGE DIAGNOSTICS, INC. AND IS NOT TO BE DISCLOSED, REPRODUCED OR USED FOR MANUFACTURE WITHOUT ITS PRIOR WRITTEN AUTHORITY.		TITLE MOUNT, MONITOR, SUSPENSION, 1000F-4		
COPYRIGHT ©		IDI		
TOLERANCES UNLESS OTHERWISE SPECIFIED FRACTION XX ± 1/32 DECIMAL .XX ± .01 DECIMAL .XXX ± .005 ANGULAR ° ± 1°		DRAWN BY MDL	DATE 1/11/2019	MATERIAL 35.26lbs.
APPROVED BY SEE ECO		DATE		PART NUMBERS A240-884
Image Diagnostics, Inc. idi		SCALE 1:4		FINISH ASSEMBLY
310 Authority Drive Fitchburg, MA 01420		SIZE D	DWG NO. A240-884	NEXT LEVEL A240-1407
				REV B