


ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	C000-0044	BUMPER, RUBBER, BLACK - W/METAL INSERT, 1" DIA X 1" LG	4
2	C000-0046	BRG, TH, 3/8 X 3/4 X 1/16	2
3	C000-0533	NUT, T-SLOT, HD, M6	14
4	C000-0587	BRG, FL .375 X .50 X .3125	2
5	C000-0588	SCR, SOC, SHLDR, SST, 3/8 X 5/8	2
6	C000-0626	SPACER, SHAFT, .375 X .563 X .031	2
7	C000-0687	RIVET, PUSH, PLASTIC, BLACK	2
8	C000-0706	BUMPER, URETHANE, W/INSERT, 3/4" DIA X 3/4" LG	1
9	C000-1007	STNDF, M-F, 10-32 X 1-1/2 X 3/8 HEX	4
10	C000-1350	WSHR, FLT, NYLON, #12, .031x.25x.375	8
11	J000-0007	BLOCK, TERMINAL, 12POS.	2
12	J000-0056	TIE WRAP ANCHOR	12
13	P240-762	BRACKET, MONITOR, PAINTED	6
14	P240-783	COVER, ARRAY, LEFT, 23IN. (PT)	2
15	P240-785	COVER, ARRAY, RIGHT, 23IN. (PT)	2
16	P240-844	SUPPORT PLATE, BI-LEVEL SUSPENSIONS (PT)	1
17	P240-846	COVER, REAR, BI-LEVEL SUSPENSION (PT)	1
18	P240-850	COVER, WIREWAY, SUSPENSIONS (PT)	1
19	P240-860	PLATE, METER BRACKET, PAINTED	1
20	P240-863	HANDLE, S3.25IN., SUSPENSIONS	1
21	P240-865	CAP, HANDLE, LEFT (PT)	1
22	P240-867	CAP, HANDLE, RIGHT (PT)	1
23	P240-874	HANDLE, MOUNT, RIGHT, WELDMENT (PT)	1
24	P240-875	HANDLE, MOUNT, LEFT, WELDMENT (PT)	1
25	P240-914	PIVOT, TOP PLATE, MACHINED (PT)	1
26	Q240-816	PIVOT, MOUNT, MONITOR, (CA), (CENTER POSITION)	2
27	Q240-818	PIVOT, MOUNT, MON, LEFT (CA)	2
28	Q240-873	NUT, T-SLOT, M5 X 1-1/4, EN	6
29	Q240-876	CAP, SWIVEL, SUSPENSIONS (EN)	4
30	Q240-893	PIVOT, MOUNT, MON, RIGHT (CA)	2
31	Q240-897	NUT, T-SLOT, M5 X 1-1/2, EN	2
32	Q240-1045	RING, THRUST (MO) 3.386 X 4.063 X .125	1
33	Q240-1049	CAP, ROTATION, (CA), 1000F	1
34	X240-1017	PAD, PROTECTIVE, SUSPENSION	1
35	Z240-814	SHIM, .188 X 1 X 5.625, SUSPENSIONS	2
36	Z240-851	BUSHING, SWIVEL, SUSPENSIONS	4
37	Z240-872	CROSSBAR, MONITOR MOUNT, 43IN.	2
38	Z240-1046	RING, THRUST, 3.386 X 4.063 X .020, 1000F	1
39	Z240-1047	BUSHING, PIVOT, 1000F	1
40	FLOOR STOCK	SCR, PHH, PAN, ZP, #8-32 X 1/4	12
41	FLOOR STOCK	WSHR, LCK, INT, ZP, #10	2
42	FLOOR STOCK	NUT, HEX, K-TYPE, BR, #10-32	4
43	FLOOR STOCK	NUT, HEX, K-TYPE, ZP, #10-32	2
44	FLOOR STOCK	NUT, NYLOK, 5/16-18	2
45	FLOOR STOCK	SCR, SLT, RND, BRS, #10-32 X 1	2
46	FLOOR STOCK	SCR, SHCS, SS, 6MM X 25MM	6
47	FLOOR STOCK	SCR, SHCS, SS, 6MM X 20MM	6
48	FLOOR STOCK	SCR, SHCS, ALY, #6-32 X 5/8	6
49	FLOOR STOCK	SCR, SOC, BTN, SST, #10-32 X 3/8	10
50	FLOOR STOCK	SCR, SHCS, ALY, #10-32 X 5/8	4
51	FLOOR STOCK	SCR, SHCS, ALY, #10-32 X 3/4	7
52	FLOOR STOCK	WSHR, LCK, SPL, ZP, #10	11
53	FLOOR STOCK	SSCR, SOC, CUP, ALY, #10-32 X 5/16	6
54	FLOOR STOCK	SCR, SHCS, SS, 5MM X 35MM	8
55	FLOOR STOCK	SCR, SOC, BTN, SST, M6 X 12MM	8
56	FLOOR STOCK	SCR, SHCS, SS, 5MM X 16MM	4
57	FLOOR STOCK	SCR, SHCS, SST, 1/4-20 X 3/4	2
58	FLOOR STOCK	SSCR, SOC, CUP, SS, #10-32 X 3/16	2
59	FLOOR STOCK	SCR, SHCS, SS, 5MM X 25MM	4

A240-932 NOTES:
1. APPLY LOCTITE TO ALL THREADED FASTENERS UNLESS OTHERWISE NOTED.
2. FOR LOCATIONS AND QUANTITIES OF LABELING, REFER TO IDI DRAWING #A240-1402.

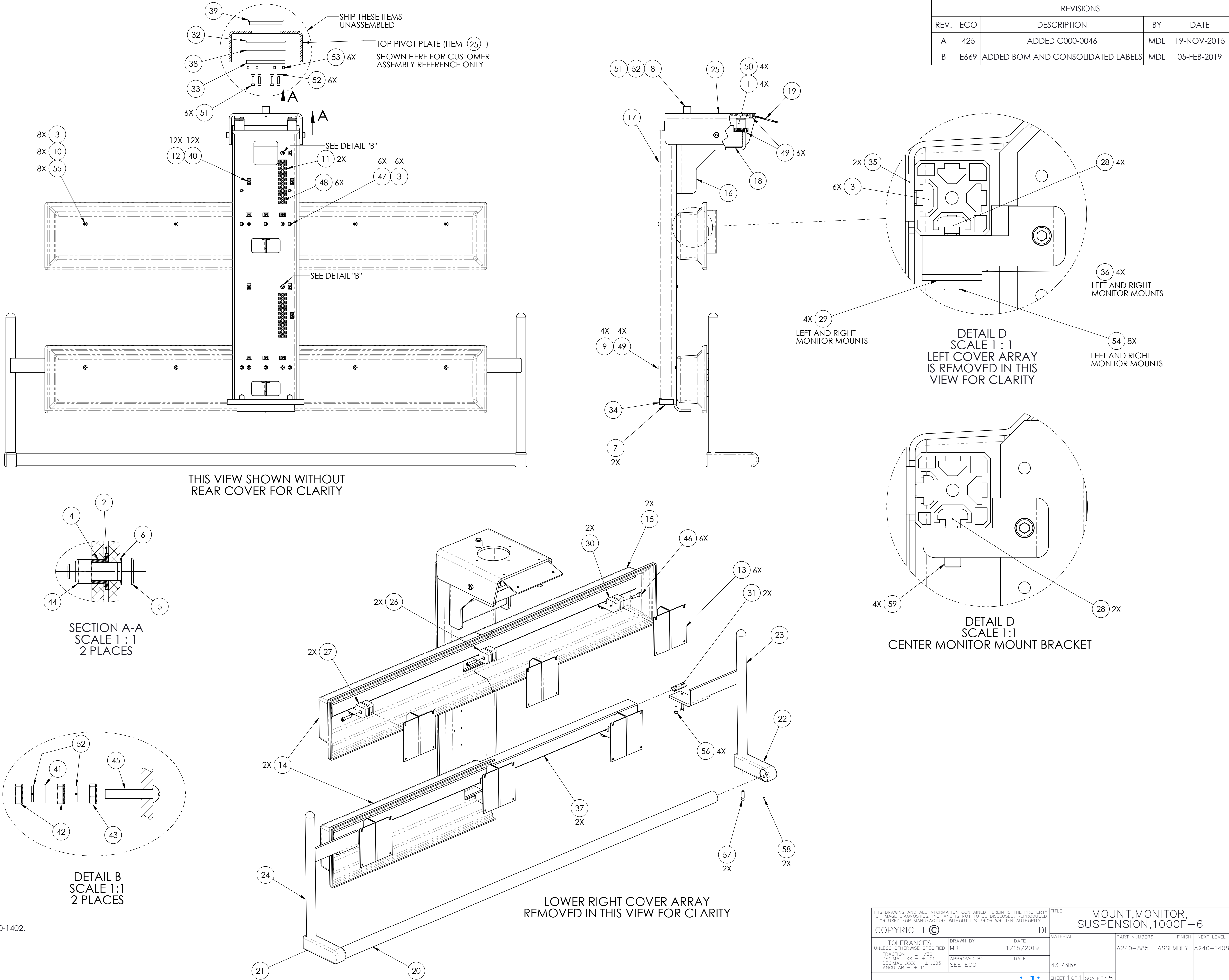

REVISIONS				
REV.	ECO	DESCRIPTION	BY	DATE
A	425	ADDED C000-0046	MDL	19-NOV-2015
B	E669	ADDED BOM AND CONSOLIDATED LABELS	MDL	05-FEB-2019

THIS DRAWING AND ALL INFORMATION CONTAINED HEREIN IS THE PROPERTY OF IMAGE DIAGNOSTICS, INC. AND IS NOT TO BE DISCLOSED, REPRODUCED OR USED FOR MANUFACTURE WITHOUT ITS PRIOR WRITTEN AUTHORITY.		TITLE MOUNT, MONITOR, SUSPENSION, 1000F-6		
COPYRIGHT © IDI		MATERIAL	PART NUMBERS	FINISH
TOLERANCES UNLESS OTHERWISE SPECIFIED FRACTION $\pm 1/32$ DECIMAL $\pm .01$ DECIMAL $\pm .005$ ANGULAR $\pm 1^\circ$		43.73lbs.	A240-885	ASSEMBLY
DRAWN BY MDL		DATE 1/15/2019	NEXT LEVEL A240-1408	
APPROVED BY SEE ECO		DATE	SCALE 1:5	
Image Diagnostics, Inc. idi		SIZE D	DWG NO. A240-885	REV B
310 Authority Drive Fitchburg, MA 01420				