


ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	C000-0044	BUMPER, RUBBER, BLACK - W/METAL INSERT, 1" DIA X 1"LG	4
2	C000-0046	BRG, TH, 3/8 X 3/4 X 1/16	2
3	C000-0533	NUT, T-SLOT, HD, M6	3
4	C000-0587	BRG,FL .375 X .50 X .3125	2
5	C000-0588	SCR,SOC,SHLDR,SST,3/8 X 5/8	2
6	C000-0626	SPACER,SHAFT,.375 X .563 X .031	2
7	C000-0687	RIVET,PUSH,PLASTIC,BLACK	2
8	C000-0706	BUMPER, URETHANE, W/INSERT, 3/4" DIA X 3/4" LG	1
9	C000-0770	NUT,STRUT,#10-32 WITHOUT SPRING	2
10	C000-1007	STNDF,M-F,10-32 X 1-1/2 X 3/8 HEX	2
11	J000-0007	BLOCK, TERMINAL, 12POS.	1
12	J000-0056	TIE WRAP ANCHOR	6
13	P240-762	BRACKET, MONITOR, PAINTED	1
14	P240-797	SUPPORT PLATE, SUSPENSIONS (PT)	1
15	P240-848	COVER,REAR,SUSPENSION (PT)	1
16	P240-850	COVER,WIREWAY,SUSPENSIONS (PT)	1
17	P240-860	PLATE, METER BRACKET, PAINTED	1
18	P240-914	PIVOT, TOP PLATE, MACHINED (PT)	1
19	Q240-816	PIVOT, MOUNT, MONITOR, (CA), (CENTER POSITION)	1
20	Q240-873	NUT, T-SLOT, M5 X 1-1/4, EN	1
21	Q240-1045	RING, THRUST (MO) 3.386 X 4.063 X .125	1
22	Q240-1049	CAP, ROTATION, (CA), 1000F	1
23	X240-1017	PAD, PROTECTIVE, SUSPENSION	1
24	Z240-814	SHIM, .188 X 1 X 5.625, SUSPENSIONS	1
25	Z240-927	HANDLE, 1.5IN., SUSPENSION	1
26	Z240-928	CROSSBAR, MONITOR MOUNT, 6IN.	1
27	Z240-1046	RING, THRUST, 3.386 X 4.063 X .020, 1000F	1
28	Z240-1047	BUSHING, PIVOT, 1000F	1
29	FLOOR STOCK	SCR,PHH,PAN,ZP,#8-32 X 1/4	6
30	FLOOR STOCK	WSHR,LCK,INT,ZP,#10	1
31	FLOOR STOCK	NUT,HEX,K-TYPE,BR,#10-32	2
32	FLOOR STOCK	NUT,HEX,K-TYPE,ZP,#10-32	1
33	FLOOR STOCK	HEX NUT, NY-LOCK, #10-32, ZINC	4
34	FLOOR STOCK	NUT,NYLOK,5/16-18	2
35	FLOOR STOCK	SCR,SLT,RND,BRS,#10-32 X 1	1
36	FLOOR STOCK	SCR,SOC,FLT,ALY,#10-32 X 1 1/4	2
37	FLOOR STOCK	SCR,SOC,FLT,ALY,#10-32 X 1	2
38	FLOOR STOCK	SCR,SHCS,SS,6MM X 25MM	1
39	FLOOR STOCK	SCR,SHCS,SS,5MM X 25MM	2
40	FLOOR STOCK	SCR,SHCS,SS,6MM X 20MM	3
41	FLOOR STOCK	SCR,SHCS,ALY,#6-32 X 5/8	3
42	FLOOR STOCK	SCR,SOC,BTN,SST,#10-32 X 3/8	8
43	FLOOR STOCK	SCR,SHCS,ALY,#10-32 X 5/8	4
44	FLOOR STOCK	SCR,SHCS,ALY,#10-32 X 3/4	7
45	FLOOR STOCK	WSHR,LCK,SPL,ZP,#10	9
46	FLOOR STOCK	SSCR,SOC,CUP,ALY,#10-32 X 5/16	6

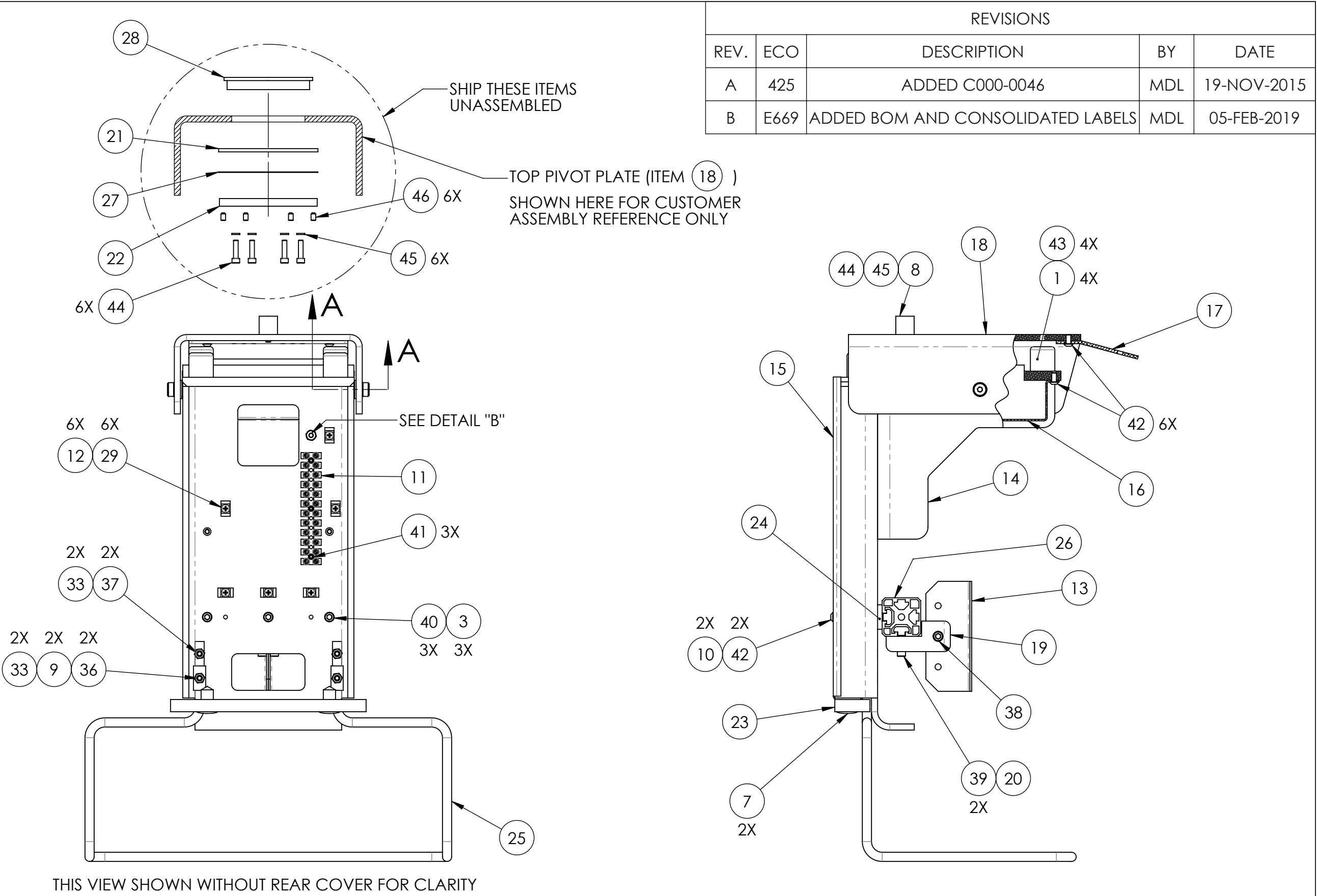

- A240-930 NOTES:
1. APPLY LOCTITE TO ALL THREADED FASTENERS UNLESS OTHERWISE NOTED.
 2. FOR LOCATIONS AND QUANTITIES OF LABELING, REFER TO IDI DRAWING #A240-1402.

THIS DRAWING AND ALL INFORMATION CONTAINED HEREIN IS THE PROPERTY OF IMAGE DIAGNOSTICS, INC. AND IS NOT TO BE DISCLOSED, REPRODUCED OR USED FOR MANUFACTURE WITHOUT ITS PRIOR WRITTEN AUTHORITY				TITLE MOUNT, MONITOR, SUSPENSION, 1000F-1		
COPYRIGHT © IDI				MATERIAL		
TOLERANCES UNLESS OTHERWISE SPECIFIED FRACTION = ± 1/32 DECIMAL .XX = ± .01 DECIMAL .XXX = ± .005 ANGULAR = ± 1°		DRAWN BY MDL		DATE 1/7/2019		PART NUMBERS A240-930
		APPROVED BY SEE ECO		DATE		FINISH ASSEMBLY
				NEXT LEVEL A240-1404		
				14.93lbs.		
Image Diagnostics, Inc. 				SHEET 1 OF 1		
				SCALE 1: 4		
310 Authority Drive Fitchburg, MA 01420				SIZE C	DWG NO. A240-930	REV B