


ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	C000-0044	BUMPER, RUBBER, BLACK - W/METAL INSERT, 1" DIA X 1"LG	4
2	C000-0046	BRG, TH, 3/8 X 3/4 X 1/16	2
3	C000-0533	NUT, T-SLOT, HD, M6	7
4	C000-0587	BRG,FL .375 X .50 X .3125	2
5	C000-0588	SCR,SOC,SHLDR,SST,3/8 X 5/8	2
6	C000-0626	SPACER,SHAFT,.375 X .563 X .031	2
7	C000-0687	RIVET,PUSH,PLASTIC,BLACK	2
8	C000-0706	BUMPER, URETHANE, W/INSERT, 3/4" DIA X 3/4" LG	1
9	C000-0770	NUT,STRUT,#10-32 WITHOUT SPRING	2
10	C000-1007	STNDF,M-F,10-32 X 1-1/2 X 3/8 HEX	2
11	C000-1350	WSHR,FLT,NYLON,#12,.031x.25x.375	4
12	J000-0007	BLOCK, TERMINAL, 12POS.	1
13	J000-0056	TIE WRAP ANCHOR	6
14	P240-762	BRACKET, MONITOR, PAINTED	2
15	P240-793	COVER,ARRAY,LEFT,14IN. (PT)	1
16	P240-795	COVER,ARRAY,RIGHT,14IN. (PT)	1
17	P240-797	SUPPORT PLATE, SUSPENSIONS (PT)	1
18	P240-848	COVER,REAR,SUSPENSION (PT)	1
19	P240-850	COVER,WIREWAY,SUSPENSIONS (PT)	1
20	P240-860	PLATE, METER BRACKET, PAINTED	1
21	P240-914	PIVOT, TOP PLATE, MACHINED (PT)	1
22	Q240-818	PIVOT, MOUNT, MON, LEFT (CA)	1
23	Q240-873	NUT, T-SLOT, M5 X 1-1/4, EN	2
24	Q240-876	CAP, SWIVEL, SUSPENSIONS (EN)	2
25	Q240-893	PIVOT, MOUNT, MON, RIGHT (CA)	1
26	Q240-1045	RING, THRUST (MO) 3.386 X 4.063 X .125	1
27	Q240-1049	CAP, ROTATION, (CA), 1000F	1
28	X240-1017	PAD, PROTECTIVE, SUSPENSION	1
29	Z240-814	SHIM, .188 X 1 X 5.625, SUSPENSIONS	1
30	Z240-851	BUSHING, SWIVEL, SUSPENSIONS	2
31	Z240-871	CROSSBAR, MONITOR MOUNT, 25IN.	1
32	Z240-927	HANDLE, 15IN., SUSPENSION	1
33	Z240-1046	RING, THRUST, 3.386 X 4.063 X .020, 1000F	1
34	Z240-1047	BUSHING, PIVOT, 1000F	1
35	FLOOR STOCK	SCR,PHH,PAN,ZP,#8-32 X 1/4	6
36	FLOOR STOCK	WSHR,LCK,INT,ZP,#10	1
37	FLOOR STOCK	NUT,HEX,K-TYPE,BR,#10-32	2
38	FLOOR STOCK	NUT,HEX,K-TYPE,ZP,#10-32	1
39	FLOOR STOCK	HEX NUT, NY-LOCK, #10-32, ZINC	4
40	FLOOR STOCK	NUT,NYLOK,5/16-18	2
41	FLOOR STOCK	SCR,SLT,RND,BRS,#10-32 X 1	1
42	FLOOR STOCK	SCR,SOC,FLT,ALY,#10-32 X 1 1/4	2
43	FLOOR STOCK	SCR,SOC,FLT,ALY,#10-32 X 1	2
44	FLOOR STOCK	SCR,SHCS,SS,6MM X 25MM	2
45	FLOOR STOCK	SCR,SHCS,SS,6MM X 20MM	3
46	FLOOR STOCK	SCR,SHCS,ALY,#6-32 X 5/8	3
47	FLOOR STOCK	SCR,SOC,BTN,SST,#10-32 X 3/8	8
48	FLOOR STOCK	SCR,SHCS,ALY,#10-32 X 5/8	4
49	FLOOR STOCK	SCR,SHCS,ALY,#10-32 X 3/4	7
50	FLOOR STOCK	WSHR,LCK,SPL,ZP,#10	9
51	FLOOR STOCK	SSCR,SOC,CUP,ALY,#10-32 X 5/16	6
52	FLOOR STOCK	SCR,SHCS,SS,5MM X 35MM	4
53	FLOOR STOCK	SCR,SOC,BTN,SST,M6 X 12MM	4

A240-932 NOTES:
1. APPLY LOCTITE TO ALL THREADED FASTENERS UNLESS OTHERWISE NOTED.
2. FOR LOCATIONS AND QUANTITIES OF LABELING, REFER TO IDI DRAWING #A240-1402.

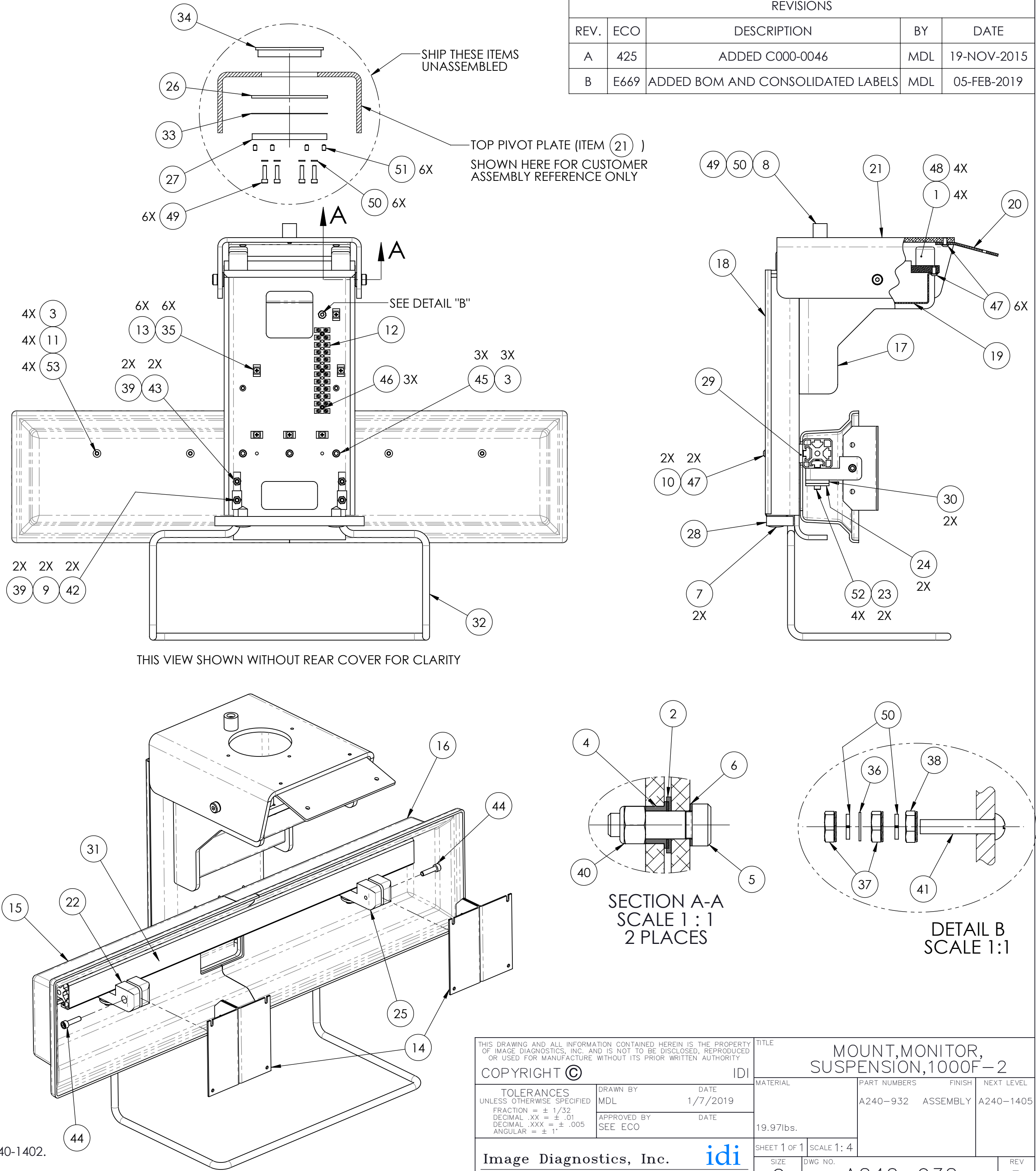

REVISIONS				
REV.	ECO	DESCRIPTION	BY	DATE
A	425	ADDED C000-0046	MDL	19-NOV-2015
B	E669	ADDED BOM AND CONSOLIDATED LABELS	MDL	05-FEB-2019

THIS DRAWING AND ALL INFORMATION CONTAINED HEREIN IS THE PROPERTY OF IMAGE DIAGNOSTICS, INC. AND IS NOT TO BE DISCLOSED, REPRODUCED OR USED FOR MANUFACTURE WITHOUT ITS PRIOR WRITTEN AUTHORITY				TITLE MOUNT, MONITOR, SUSPENSION, 1000F-2		
COPYRIGHT © IDI		DRAWN BY MDL DATE 1/7/2019		MATERIAL		PART NUMBERS
TOLERANCES UNLESS OTHERWISE SPECIFIED FRACTION = ± 1/32 DECIMAL .XX = ± .01 DECIMAL .XXX = ± .005 ANGULAR = ± 1°		APPROVED BY SEE ECO DATE		19.97lbs.		FINISH
Image Diagnostics, Inc. 		310 Authority Drive Fitchburg, MA 01420		SHEET 1 OF 1 SCALE 1:4		NEXT LEVEL
SIZE C		DWG NO. A240-932		REV B		A240-1405