



QUICK GUIDE

X-ray OP Table, medifa 8000, Model 800200

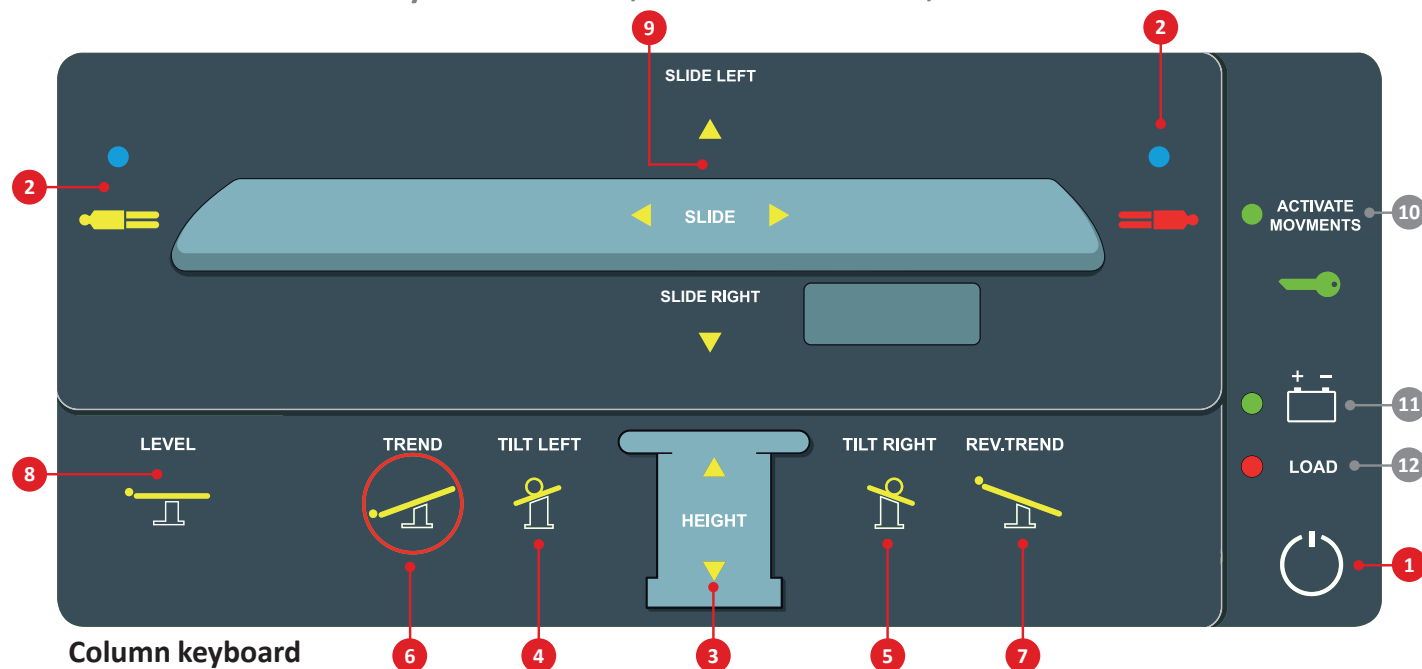

Column keyboard

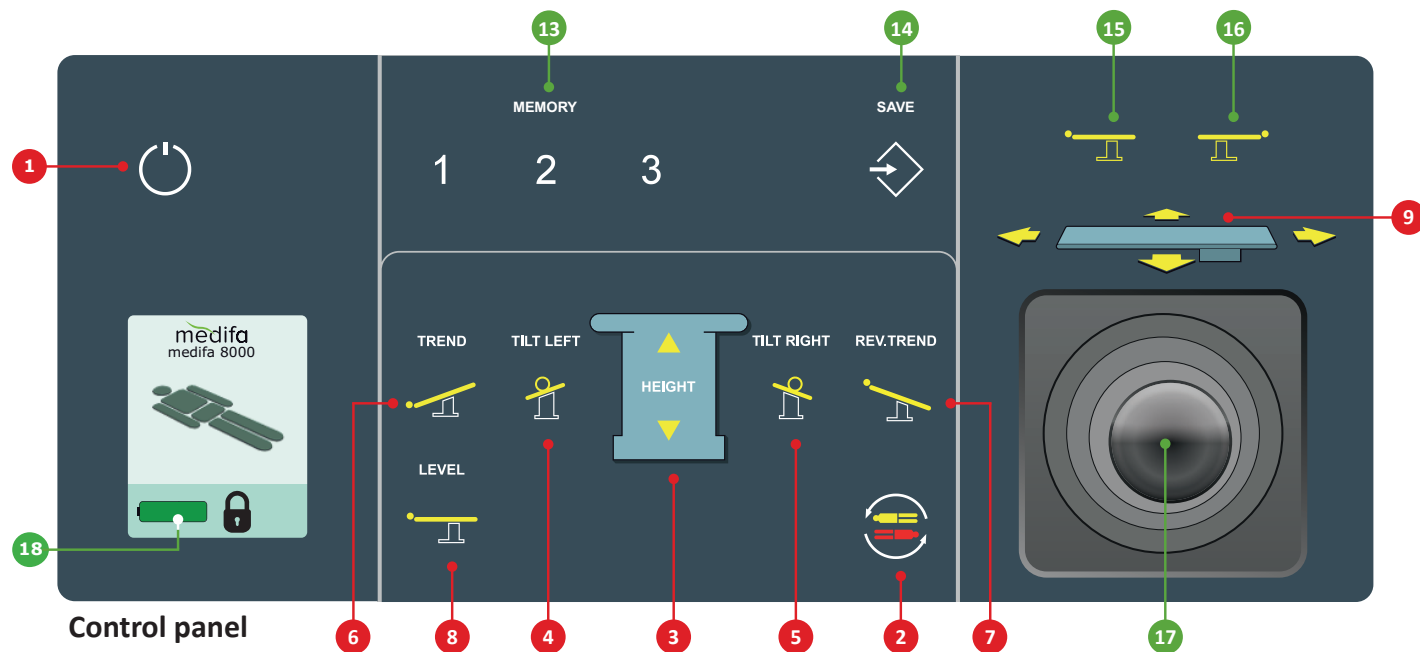

Control panel

Column keyboard only



- 10 ACTIVATE MOVEMENTS
Key lock on/off
- 11 Battery status
- 12 LOAD (charging status)

Column keypad + control panel

- 1 On/off switch
- 2 Normal/Reverse
- 3 HEIGHT (up/down)
- 4 TILT LEFT (lateral left)
- 5 TILT RIGHT (lateral right)
- 6 TREND (Trendelenburg)
- 7 REV. TREND (reverse Trendelenburg)
- 8 LEVEL (home position)
- 9 SLIDE (lengthwise shift / sideways shift)

Control panel only

- 13 MEMORY function
- 14 SAVE
- 15 User position (right)
- 16 User position (left)
- 17 Joystick
- 18 Battery status

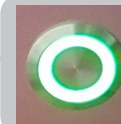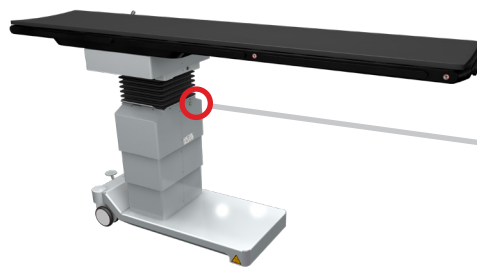

SWITCH ON

- For operating

SWITCH OFF

- Before each cleaning
- During a long pause



ATTENTION

This quick guide is only a part of the X-ray OP table's operating instructions. This guide alone, without the operating instructions manual, is NOT sufficient to ensure the safe use of this medifa X-ray OP table. Please read the operating instructions manual for your medifa X-ray OP table FIRST. Please pay particular attention to the safety instructions, explanations of the warning labels, as well as the proper and intended usage sections contained therein.

MOVING THE X-RAY OP TABLE

The X-ray OP table may only be moved or driven using two persons and when it is at its lowest height!

- 1 The X-ray OP table must be adjusted to its movement position before it is moved or driven:
 - Move to the zero position
 - Fold back or remove any accessories
 - Set the table plate to the lowest position
 - Disconnect the mains power plug and the equipotential bonding cable
- 2 Release the brake and move the table manually.

Do not let go while moving the table. Stop the table manually.

BRAKING THE X-RAY OP TABLE

Do not use the brake while you are moving the OP table!

First stop the X-ray OP table:

- 1 Stop the X-ray OP table.
- 2 Then press the brake pedal down.

CHARGING THE X-RAY OP TABLE

Do not use an extension cable!

- 1 Display during operation:

 Normal operations

Recharge the X-ray OP table immediately whenever the display changes to:

 **0-20% of charge capacity**

- 2 Connect the power cable. The symbol on the control panel changes to GREEN. There is no charge level indicator on the control panel. The charging time for a full charge is about 9 hours. The column keypad displays a red LED next to the LOAD icon during the entire charging time.
- 3 Disconnect the X-ray OP table from the mains power after the specified charging time. The X-ray OP table is now ready to be used. Switch off the X-ray OP table during long periods of non-use.