

Teguar Cleaning Q-Card



- Turn off monitor and unplug power cord.
- Take care not to damage or scratch the front filter or LCD panel.
- Be careful with rings or other jewelry that can touch the front filter.
- Do **NOT** apply excessive pressure on the monitor screen. Use less than 1N (0.2lbs) of force.
- Do **NOT** apply or spray liquid directly to the front filter, panel or cabinet as excess liquid may cause damage to internal electronics. Instead, apply the liquid to the cleaning cloth.
- Cleaners could cause serious damage if it leaks inside the monitor while cleaning.
- Follow your hospital or clinical protocol for the handling of blood and body fluids.
- The display is not disinfected nor packed in sterile environment.
- Follow your hospital or clinical protocol in instances that need the display disinfected prior to installation.
- Cleaning guidelines for displays must only be carried out by trained medical professionals and must not be handled by patients.
- For complete cleaning instructions please reference the monitor's user manual. An electronic copy of the manual is available upon request through IDI.
- Using a spray applicator, apply any of the approved liquids from the list onto a cloth then apply cloth to unit.

Teguar Monitors Approved Cleaners



- Mild soap-and-water solution
- Clorox Disinfecting Wipes
- Clorox Healthcare Bleach Germicidal Wipes
- Clorox Commercial Solutions Hydrogen Peroxide Cleaner Disinfectant Wipes
- Lonza Disinfectant Wipes
- Lysol Brand Clean & Fresh Multi Surface Cleaner (20% cleaner solution to water ratio)
- Purell Professional Surface Disinfectant Wipes
- Sani-Cloth Prime Germicidal Disposable Wipes

Do Not Use List



- Alcohol at concentration > 70%
- Methyl or Ethyl > 35%
- Thinner or Benzine
- Strong alkali lyes
- Strong solvents
- Acids
- Detergents with fluoride
- Detergents with ammonia at concentration > 1.6%
- Abrasive cleaners
- Detergents with abrasives
- Steel wool
- Sponges with abrasives
- Steel blades
- Cloth with steel threads
- 409