



INSERT STRAP INTO PAD AFTER  
LOOPING AROUND ONE RAIL

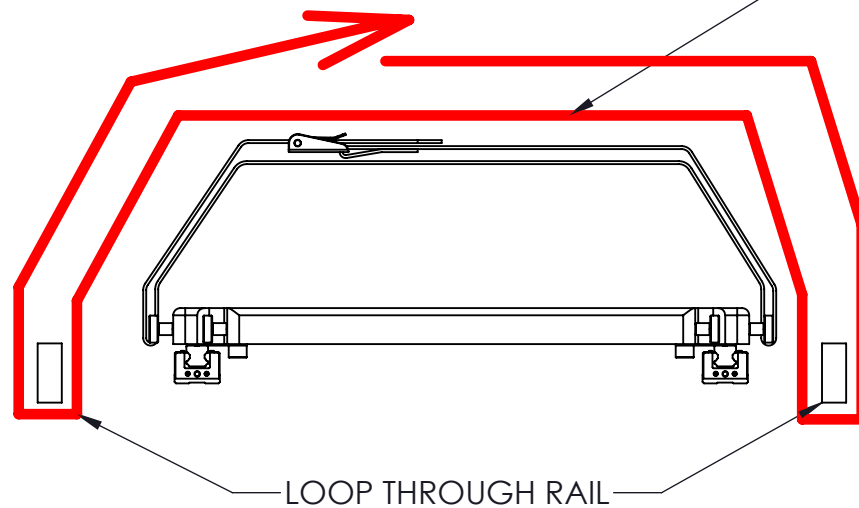


**SETUP OPTION 1 STRAP WITH PAD**

WHEN RAIL IS AVAILABLE TO ATTACH  
TO, LOOP STRAP AROUND ONE RAIL,  
SLIDE STRAP THROUGH SLEEVE OF  
PAD, THEN GO BACK OVER PATIENT,  
LOOP AROUND OPPOSITE SIDE RAIL,  
THEN PULL TIGHT TO SECURE PATIENT

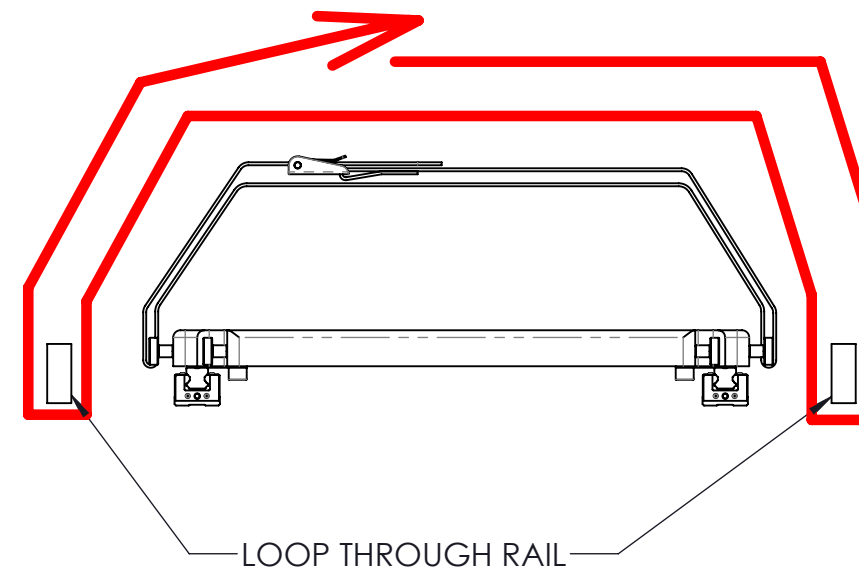


**SETUP OPTION 2 (STRAP ONLY NO PAD)**

WHEN RAIL IS AVAILABLE TO ATTACH TO,  
LOOP STRAP THROUGH AROUND RAIL,  
THEN PULL BACK OVER PATIENT, LOOP  
AROUND OPPOSITE SIDE RAIL, THEN PULL  
TIGHT TO SECURE PATIENT.

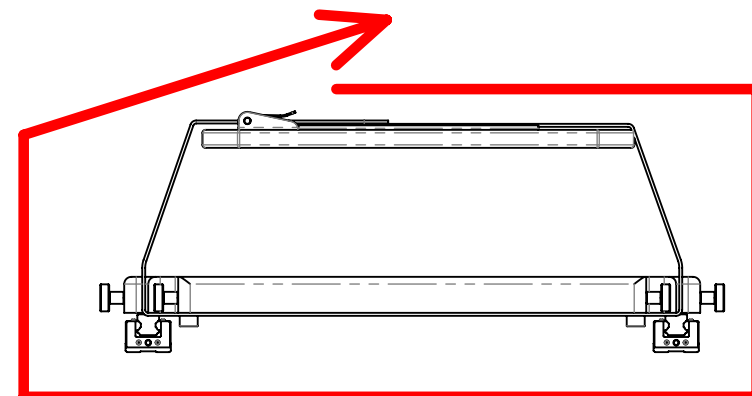

**SETUP OPTION 3**

WHEN NO RAIL IS AVAILABLE TO  
ATTACH TO, SLIDE PAD ONTO  
STRAP AND WRAP ENTIRE STRAP  
AROUND THE TABLE

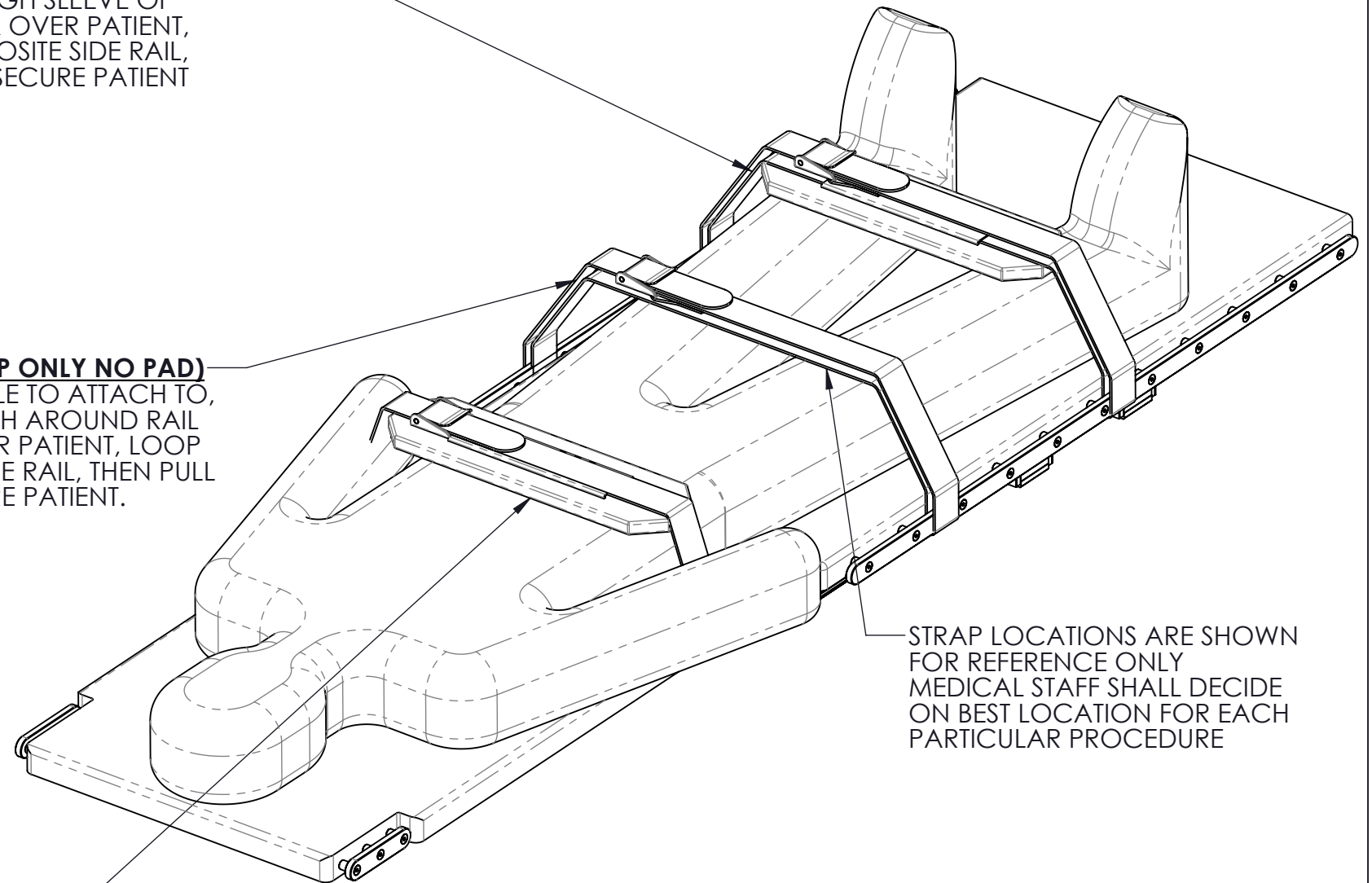

STRAP LOCATIONS ARE SHOWN  
FOR REFERENCE ONLY  
MEDICAL STAFF SHALL DECIDE  
ON BEST LOCATION FOR EACH  
PARTICULAR PROCEDURE

STRAP KIT A100-3731 INCLUDES:  
QTY 2 OF PN# X100-3705 STRAP  
QTY 1 OF PN# X100-3710 PAD

Image Diagnostics, Inc. **idi**  
310 Authority Drive Fitchburg, MA 01420

SHEET 1 OF 1  
SIZE  
B

TITLE PATIENT RESTRAINT STRAP INSTRUCTIONS		REV A
DWG NO. L100-1039		