

# Aspect 100-4 Q-CARD



**WARNING! Read operator manual prior to use**

## SETUP/PATIENT PREPARATION:

Moving table (NEVER WITH PATIENT ON BOARD):

- 1) Unlock Rear Casters using Caster Lock Pedal.
- 2) Unlock Front Steering using Front Steer Lock Pedal.
- 3) Position table to desired location.
- 4) Secure table mobility by locking Casters and front steering using Rear Caster Lock Pedal and Front Steer Lock Pedal.

AC Power:

- 5) Plug power cord into properly grounded outlet.
- 6) Protect power cord from damage by casters or other equipment.

Controls:

- 1) Emergency Stop Pushbutton disables all tabletop motion control when engaged.
- 2) If tabletop motion controls fail to operate, make sure that the Emergency Stop Pushbutton is NOT pushed in.

## BATTERY OPERATION (DC POWER):

- 1) The table is FULLY functional, but motions are slower.
- 2) Connect to AC power for a minimum of 8 hours daily to ensure a proper battery charge.
- 3) Failure to use motorized motion feature of table for more than 1-1/2 days will cause the motion controller to be in stop mode. Make sure power cord is plugged in, push in Emergency Stop Pushbutton and disengage to restore motion capabilities.



### CPR:


Return table to level, retract tabletop to minimize overhang and lower table position to a comfortable height to perform CPR.

## BRAKES:

The 100-4 table braking system utilizes a passive magnetic brake; it remains engaged in the event of a complete loss of electrical power. Power is necessary to release brakes.

Braking force is limited by the magnet strength. Use caution when applying pressure to the side of the table as the brakes may be overcome if more than 50 pounds of force is applied.

To reduce potential motion during patient transfers with larger patients, release the table brake and move the top in the direction the patient will be pulled during loading or unloading until the tabletop transverse travel limit is reached. Release the panning handle to re-engage the lock and then transfer the patient.

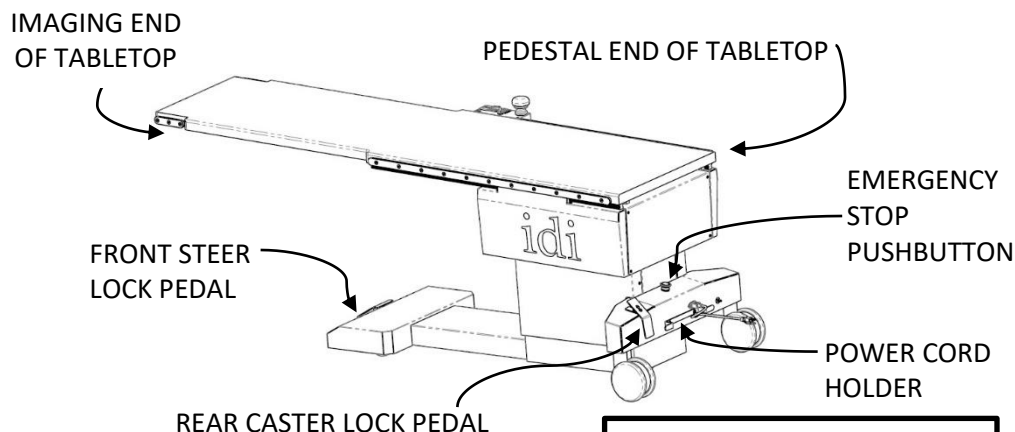

## CLEANING AND STORAGE:

- 1) Unplug power cord and wrap it around the cord holder.
- 2) Push in Emergency Stop Pushbutton.
- 3) Lock rear casters and front steering.
- 4) Clean and disinfect table.




Scan to  
Access  
User  
Manuals  
Online

# Aspect 100-4 Q-CARD

## TABLETOP MOTION CONTROL PANHANDLE

### LOCK

PRESS FORWARD ON TOP HALF  
OF PEDAL TO LOCK ALL  
MOVEMENT

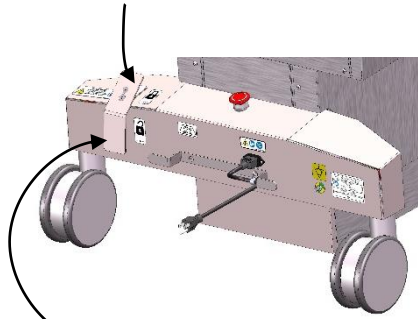

### UNLOCK


PRESS DOWN ON BOTTOM HALF  
OF PEDAL TO ALLOW REAR  
CASTERS TO SWIVEL FREELY



## CASTER LOCK/STEER PEDAL POSITIONS

PANHANDLE  
KNOB/ TABLETOP  
FLOAT BRAKE  
(Lift underside of  
Panhandle Knob to  
float tabletop. It will  
not operate if  
tabletop is not level)

Handle to clamp Panhandle  
onto Accessory Rail  
(clockwise to unlock &  
counterclockwise to lock)

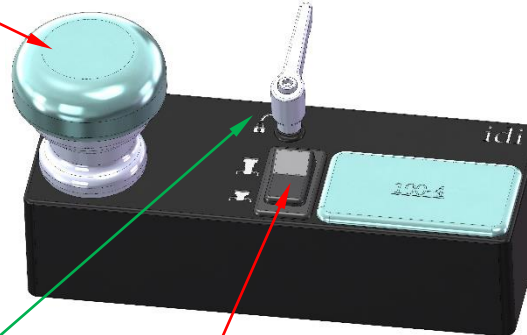

ELEVATE  
SWITCH

**STEER LOCK**  
PRESS DOWN ON PEDAL TO  
RESTRICT FRONT MOVEMENT TO  
FORWARD AND BACKWARDS ONLY


**FLOAT POSITION**  
PULL UP ON PEDAL TO ENABLE  
MOVEMENT OF FRONT IN ALL  
DIRECTIONS



EMERGENCY STOP PUSHBUTTON

- PUSH IN to stop ALL tabletop motion.
- TWIST to reset.

## Tabletop Motion Handset Control

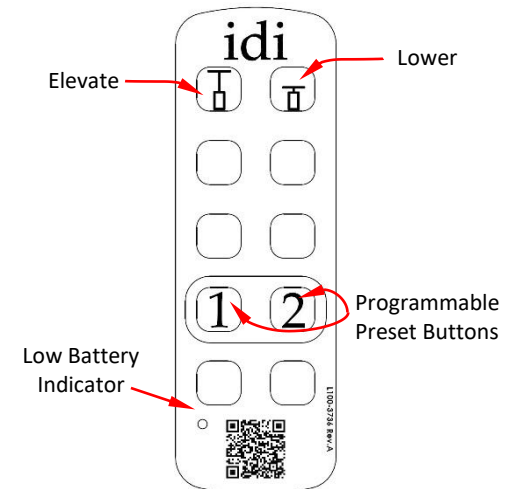

During critical procedures it is recommended that the pendant control be placed where the buttons cannot be accidentally pressed to prevent inadvertent operation or motion, causing patient injury.



Patients must be loaded from the side of table. Tilting or instability may occur if patient is loaded onto the pedestal end of table or the imaging end of the table.