

Smarter Practico Transportation Procedure

To move the table:

1. Toggle Power Switch (located below) to the "On" position.
2. Unplug the power cord located on the base of the table.





WARNING!
Read operator
manual prior to use

Scan Here for
Operator Manual
Then Scroll to
Surgical Tables



3. Using the hand pendant controller, hold down the top left button.
4. The blue light above the top right buttons will blink for a short time.
5. When you hear one long beep, both blue lights above the buttons will stay lit and the table will be ready for transportation.

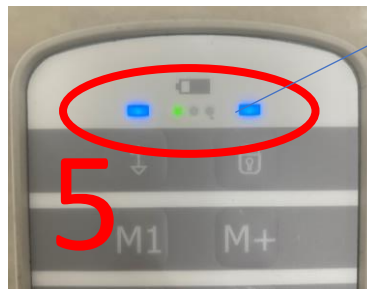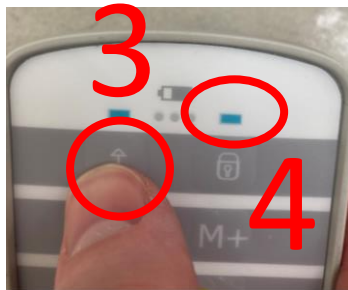

Note: if red LED is lit, plug power cord into table to recharge internal batteries, then repeat step 1.



To lock down the movement of the table:

1. Hold down the top right button until you hear one long Beep and the blue light above the lock button stays lit. (Note: wheel lock can be done with or without power cord connected to the table.)

